CareFusion

SpiroUSB

Operating Manual - English

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Contents

I.	WELCOME – English	5
	Package contents.....	6
	Installing SPCS	7
	Operation	8
	Cleaning the Transducer	9
	Servicing	10
	Consumables/ Supporting products	11
	Training	11
	Specification of the SpiroUSB	12
II.	WELCOME – CareFusion USA	14
	Package contents.....	15
	Installing SPCS	16
	Operation	17
	Cleaning the Transducer	18
	Servicing	19
	Consumables/ Supporting products	20
	Specification of the SpiroUSB	21
III.	Introduction – Français.....	22
	Contenu de l’em ballage	23
	Installation de SPCS.....	24
	Utilisation	25
	Nettoyage du spiromètre	26
	Révision.....	27
	Consommables / Produits accessoires.....	28
	Caractéristiques du SpiroUSB	29

IV.	Einführung – Deutsch	30
	Kofferinhalt	31
	Installation von SPCS	32
	Operation	33
	Reinigen des Messwandlers	34
	Service	35
	Verbrauchsgüter/Unterstützungsprodukte	36
	Schulung	36
	Specification of the SpiroUSB	37
V.	Introducción – Espanol	38
	Contenido del paquete.....	39
	Instalación del SPCS.....	40
	Funcionamiento	41
	Limpieza del transductor.....	42
	Servicio	43
	Fungibles / Productos complementarios	44
	Especificación del SpiroUSB.....	45
VI.	Introdução – Português	46
	Conteúdo da embalagem	47
	Instalação do SPCS	48
	Funcionamento	49
	Limpeza do transdutor.....	50
	Manutenção	51
	Consumíveis/productos de suporte	52
	Formação.....	52
	Especificações do SpiroUSB.....	53

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Operating manual

I. Introduction - English

The SpiroUSB is a PC connected spirometer dedicated to work with the comprehensive Spirometry PC Software (SPCS).

SPCS is a fully Windows™ compatible spirometry system that interfaces seamlessly with the SpiroUSB providing many display options and includes powerful reporting and database facilities.

SPCS has many advanced features including display of real time respiratory traces, predicted values, patient database, and the ability to carry out pre and post bronchodilator and post steroid testing.

The SpiroUSB uses the CareFusion Digital Volume Transducer, an extremely stable form of volume transducer, which measures expired air directly at B.T.P.S (Body Temperature and Pressure with Saturated water vapour) thus avoiding the inaccuracies of temperature corrections. This transducer is insensitive to the effects of condensation and temperature and avoids the need for individual calibration prior to performing a test.


Package contents

The SpiroUSB is packaged in a sturdy carrying case containing this manual and the following items (Fig.1):

1. Carefusion Digital Volume Transducer
2. SpiroUSB transducer housing.

Together with SPCS, extension cable, disposable cardboard mouthpieces and nose clip.

Please note: The SpiroUSB should only be connected to a computer that has been manufactured in accordance with EN60950 1992/1993 – ‘Safety of Information Technology Equipment including Electrical Business Equipment’.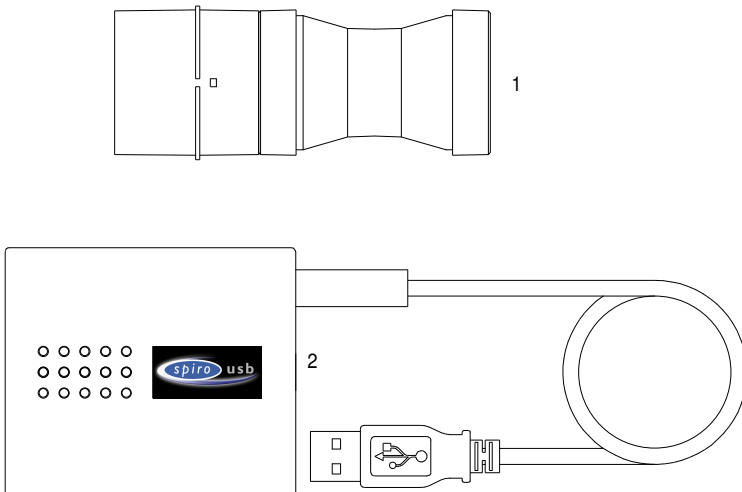

Installing SPCS

SPCS is supplied on CD ROM with your SpiroUSB. Install SPCS according to the instructions in the SPCS quick start guide.

Installing SpiroUSB device driver

The driver enables CareFusion Ltd USB devices to work with USB enabled PC software on Windows based machines.

Compatibility status

There are two versions of the USB driver. The driver used depends on the Operating System you are using. There is a driver for Windows 2000/XP Operating Systems.

The following operating systems are supported:

- Windows 2000 Professional service pack 4
- Windows XP Home Edition
- Windows XP Professional
- Windows Vista

The driver will not be supported under Windows 95, Windows 98 Gold (First Edition), Windows 98SE (Second edition), Windows ME (Millennium edition) or Windows NT 4.0 Workstation, as USB is not officially supported on these Operating Systems.

Driver installation – Windows 2000/XP/Vista

Windows 2000, Windows XP and Windows Vista support pre-installation of drivers.

If software is installed that provides support for USB, the drivers may be pre-installed, allowing automatic installation of the drivers when the device is connected to the USB port. If the device does not install automatically, follow the steps to install the device manually.

Operation

Simply connect the SpiroUSB to a spare USB port on your PC with the supplied SPCS installed. SPCS will automatically detect that the SpiroUSB has been connected to a USB port and will run, ready for you to start your spirometry session. There is no need to remove the SpiroUSB between sessions.

Please note that with certain PC configurations, even if a compatible operating system is being used, SPCS may not be launched automatically when the SpiroUSB is plugged in. In this case SPCS may be run in the usual way using the Windows on-screen Start button.

The SpiroUSB transducer is supplied with a two-meter cable. If this length is insufficient then a remote USB hub should be used. Remote USB hubs are readily available from computer accessory suppliers. For some recent computers the USB signal will work over a longer distance using an extension cable. An extension cable has been supplied for your convenience, if required. If communication problems are encountered using the extension cable then a remote USB hub must be used.

Looking after your SpiroUSB Spirometer


Please observe the following precautions:

- Avoid exposing the SpiroUSB to direct sunlight during use.
- Avoid operating the spirometer in dusty conditions or near to heating appliances or radiators.
- Do not keep the spirometer in a damp place or expose it to extremes of temperature.
- Do not direct the transducer holder towards a strong light source whilst operating the spirometer.

Cleaning the Transducer

The transducer requires no routine maintenance or servicing. However, if you wish to sterilise or clean the transducer it may be removed by means of the following procedure:

1. Rotating the turbine transducer anti-clockwise until the locating pip lines up with the small rectangular cut-out in the housing as shown below.
2. Gently pull the transducer away from the housing.
3. The transducer may now be immersed in warm soapy water for routine cleaning or immersed in a cold sterilising solution e.g. Perasafe for a maximum of 10 minutes (Alcohol and chloride solutions should be avoided). After cleaning/sterilising, the transducer should be rinsed briefly in distilled water and dried. Perasafe is available from CareFusion in convenient 81g containers, Cat No. 36-ACC1013.
4. Re-assemble the mouthpiece holder.

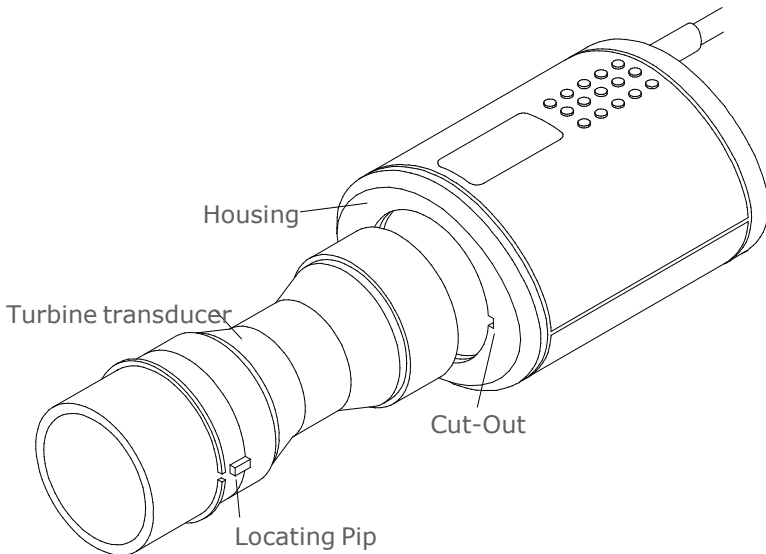Servicing

A full service manual including circuit diagrams and parts list is available upon request.

Symbols

Type B device

In accordance with Directive 93/42/EEC

Disposal in compliance with WEEE

Environment

This instrument complies with directive EN60601-1-2 electromagnetic compatibility but can be affected by cellular phones and by electromagnetic interference exceeding levels specified in EN 50082-1:1992

Electrical classification

Class I equipment.

Consumables/supporting products

Cat. No.	Description
36-PSA1000	Adult Disposable Mouthpieces (500 per box)
36-SST1000	One-way Safety Mouthpieces (500 per box)
36-PSA1200	Paediatric Disposable Mouthpieces (250 per box)
36-PSA1100	Paediatric Adaptor
36-MGF1025	MicroGard Pulmonary Filters (25 per box)
36-MGF1100	MicroGard Pulmonary Filters (100 per box)
36-ACC1013	PeraSafe Sterilising Powder 81g (to make up 5 litres of solution)
36-VOL2104	Nose Clips (pack of 5)
36-SM2125	3 Litre Calibration Syringe
36-ASS1042	SpiroUSB desktop stand

Caution

- Pulmonary filters are single patient use. If used on more than one patient there is a risk of cross-infection. Repeat use may increase air resistance and lead to an incorrect measurement.”
- Mouthpieces are single patient use. If used on more than one patient there is a risk of cross-infection. Repeat use may degrade materials and lead to an incorrect measurement.”

Training

CareFusion run spirometry training courses throughout the UK, sessions include:

- * Anatomy and Physiology of Respiratory System
- * Lung Volume Terminology
- * Objectives of Spirometry
- * Setting up the test & performing a good manoeuvre
- * Errors of technique

- * Interpretation of results
- * Using Spirometry as an aid to diagnosing airways disease
- * Choosing a Spirometer
- * Case Studies

Specification of the SpiroUSB

General

Power supply:	Input 5V 0.1A
Dimensions:	50x60x90mm.
Weight:	130g
Operating Temperature:	0 to +40° C
Operating Humidity:	30% to 90% RH
Storage Temperature:	-20 to + 70° C
Storage Humidity:	10% to 90% RH
Predicted Values:	Various – depends upon national preference.
Transducer:	Carefusion Bi-Directional Digital Volume.
Resolution:	10ml volume 0.03l/s flow
Accuracy:	+/-3%. To ATS recommendations – Standardisation of spirometry 1994 update for flows and volumes.
Measurements:	See SPCS operating manual for details of all the spirometry measurements available.

CareFusion

SpiroUSB

Operating manual

Federal (USA) law restricts this device to sale by or on the order of a physician or licensed practitioner.

CareFusion,
22745 Savi Ranch Parkway,
Yorba Linda
CA 92887-4668
USA

II. Introduction – CareFusion USA

The SpiroUSB is a PC connected spirometer dedicated to work with the comprehensive Spirometry PC Software (SPCS).

SPCS is a fully Windows™ compatible spirometry system that interfaces seamlessly with the SpiroUSB providing many display options and includes powerful reporting and database facilities.

SPCS has many advanced features including display of real time respiratory traces, predicted values, patient database, and the ability to carry out pre and post bronchodilator and post steroid testing.

The SpiroUSB uses the Digital Volume Transducer, an extremely stable form of volume transducer, which measures expired air directly at B.T.P.S (Body Temperature and Pressure with Saturated water vapour) thus avoiding the inaccuracies of temperature corrections. This transducer is insensitive to the effects of condensation and temperature and avoids the need for individual calibration prior to performing a test.

The SpiroUSB utilizes a **single patient use** disposable mouthpiece that must be disposed of after use

The SpiroUSB provides a suggested interpretation **that must be supported by clinical judgement.**



Package Contents

The SpiroUSB is packaged in a sturdy carrying case containing this manual and the following items (Fig.1):

1. Carefusion Digital Volume Transducer
2. SpiroUSB transducer housing.

Together with SPCS software, extension cable, disposable cardboard mouthpieces and nose clip.

Please note: The SpiroUSB should only be connected to a computer that has been manufactured in accordance with EN60950 1992/1993 – ‘Safety of Information Technology Equipment including Electrical Business Equipment’.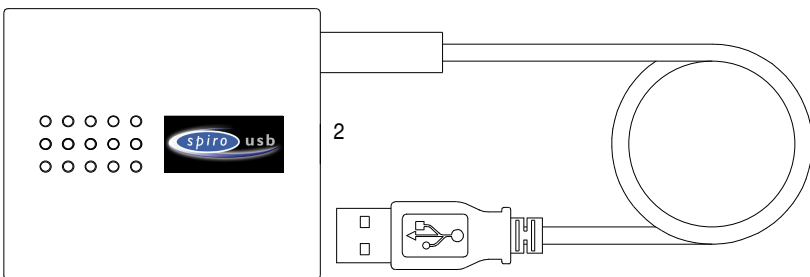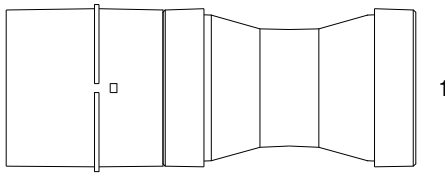

Installing SPCS

SPCS is supplied on CD ROM with your SpiroUSB. Install SPCS according to the instructions in the SPCS quick start guide.

Installing SpiroUSB Device Driver

The driver enables USB devices to work with USB enabled PC software on Windows based machines.

Compatibility Status

There are two versions of the USB driver. The driver used depends on the Operating System you are using. There is a driver for Windows 2000/XP Operating Systems.

The following operating systems are supported:

- Windows 2000 Professional
- Windows XP Home Edition
- Windows XP Professional

Driver Installation – Windows 2000/XP

Windows 2000, Windows XP and Windows Vista support pre-installation of drivers. If software is installed that provides support for USB, the drivers may be pre-installed, allowing automatic installation of the drivers when the device is connected to the USB port. If the device does not install automatically, follow the steps to install the device manually.

Operation

Simply connect the SpiroUSB to a spare USB port on your PC with the supplied SPCS software installed. SPCS will automatically detect that the SpiroUSB has been connected to a USB port and will run, ready for you to start your spirometry session. There is no need to remove the SpiroUSB between sessions.

Please note that with certain PC configurations, even if a compatible operating system is being used, SPCS may not be launched automatically when the SpiroUSB is plugged in. In this case SPCS may be run in the usual way using the Windows on-screen Start button.

The SpiroUSB transducer is supplied with a two-meter cable. If this length is insufficient then a remote USB hub should be used. Remote USB hubs are readily available from computer accessory suppliers. For some recent computers the USB signal will work over a longer distance using an extension cable. An extension cable has been supplied for your convenience, if required. If communication problems are encountered using the extension cable then a remote USB hub must be used.

Looking after your SpiroUSB Spirometer


Please observe the following precautions:

- Avoid exposing the SpiroUSB to direct sunlight during use.
- Avoid operating the spirometer in dusty conditions or near to heating appliances or radiators.
- Do not keep the spirometer in a damp place or expose it to extremes of temperature.
- Do not direct the transducer holder towards a strong light source whilst operating the spirometer.

Cleaning the Transducer

The transducer requires no routine maintenance or servicing. However, if you wish to sterilise or clean the transducer it may be removed by means of the following procedure:

1. Rotating the turbine transducer anti-clockwise until the locating pip lines up with the small rectangular cut-out in the housing as shown below.
2. Gently pull the transducer away from the housing.
3. The transducer may now be immersed in warm soapy water for routine cleaning or immersed in a cold sterilising solution e.g. Perasafe for a maximum of 10 minutes (Alcohol and chloride solutions should be avoided). After cleaning/sterilising, the transducer should be rinsed briefly in distilled water and dried.
Perasafe is available from in convenient 81g containers.
4. Re-assemble the mouthpiece holder.

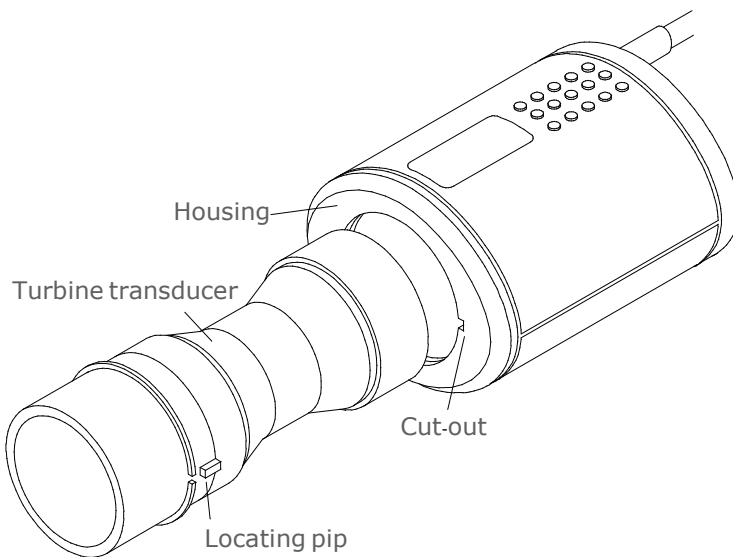Servicing

A full service manual including circuit diagrams and parts list is available upon request.

For US Customer Care: Toll Free 1-800-231-2466, phone 1-714-283-2228

Symbols

Type B device

In accordance with Directive 93/42/EEC

Disposal in compliance with WEEE

Environment

This instrument complies with directive EN60601-1-2 electromagnetic compatibility but can be affected by cellular phones and by electromagnetic interference exceeding levels specified in EN 50082-1:1992

Electrical classification

Class I equipment.

Consumables / Supporting Products

Cat. No.	Description
36-PSA1000	Adult Disposable Mouthpieces (500 per box)
36-SST1000	One-way Safety Mouthpieces (500 per box)
36-PSA1200	Paediatric Disposable Mouthpieces (250 per box)
36-PSA1100	Paediatric Adaptor
36-MGF1025	MicroGard Pulmonary Filters (25 per box)
36-MGF1100	MicroGard Pulmonary Filters (100 per box)
36-ACC1013	PeraSafe Sterilising Powder 81g (to make up 5 litres of solution)
36-VOL2104	Nose Clips (pack of 5)
36-SM2125	3 Litre Calibration Syringe
36-ASS1042	SpiroUSB desktop stand

To place an order for consumables / supporting products, or for general enquiries please contact CareFusion or your local CareFusion dealer.

Caution

- Pulmonary filters are single patient use. If used on more than one patient there is a risk of cross-infection. Repeat use may increase air resistance and lead to an incorrect measurement.”
- Mouthpieces are single patient use. If used on more than one patient there is a risk of cross-infection. Repeat use may degrade materials and lead to an incorrect measurement.”

Specification of the SpiroUSB

General

Power supply:	Input 5V 0.1A
Dimensions:	50x60x90mm.
Weight:	130g
Operating Temperature:	0 to +40° C
Operating Humidity:	30% to 90% RH
Storage Temperature:	-20 to + 70° C
Storage Humidity:	10% to 90% RH
Predicted Values:	Various – depends upon national preference.
Transducer:	Carefusion Bi-Directional Digital Volume.
Resolution:	10ml volume 0.03l/s flow
Accuracy:	+/-3%. To ATS recommendations – Standardisation of spirometry 1994 update for flows and volumes.
Measurements:	See software operating manual for details of all the spirometry measurements available.

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Customer contact information

UK Customers only

For all Sales Order processing for products, training and spare parts, Service and Technical Support enquiries, please contact the following:

CareFusion UK 232 Ltd
UK Customer Service & Support
The Crescent
Jays Close
Basingstoke
RG22 4BS

Customer Service Sales Enquiries:

Telephone: 01256 388550

Email: micro.uksales@carefusion.com

Factory Repair and Administration Enquiries:

Telephone: 01256 388552

Email: micro.ukservice@carefusion.com

Technical Support Enquiries:

Telephone: 01256 388551

Email: support.rt.eu@carefusion.com

International customers only

For all Sales Order processing for products and Spare parts, Service and Technical Support enquiries, please contact the following:

Carefusion Germany 234 GmbH
Customer Service & Support International
Leibnizstrasse 7
97204 Hoechberg
Germany

Customer Service Sales Enquiries:

Telephone: 0049 931 4972 670

Email: micro.internationalsales@carefusion.com

Factory Repair and Administration Enquiries:

Telephone: 0049 931 4972 867

Email: support.admin.eu@carefusion.com

Technical Support Enquiries:

Telephone: 0049 931 4972 127

Email: support.rt.eu@carefusion.com

CareFusion UK 232 Ltd.,
Quayside
Chatham Maritime
Kent ME4 4QY
U.K.

CE
0086

JDE no. 36-MAN1275
Drg no. 075-11WW
Issue 1.0
October 2009
© CareFusion 2009