

BTE 105

Behind-the-Ear Hearing Aids

VIRON
ZERENA



Made for

 iPhone | iPad | iPod

Instructions for Use

bernafon[®]
Your hearing • Our passion

Model overview

This booklet is valid for the following hearing aid families and models:

Viron

- Viron 9 – VN9 B 105
- Viron 7 – VN7 B 105
- Viron 5 – VN5 B 105

Zerena

- Zerena 9 – ZR9 B 105
- Zerena 7 – ZR7 B 105
- Zerena 5 – ZR5 B 105
- Zerena 3 – ZR3 B 105
- Zerena 1 – ZR1 B 105

Introduction to this booklet

This booklet shows you how to use and maintain your new hearing aid. Please read the booklet carefully including the warning sections. This will help you to get the most out of your new hearing aid.

Your hearing care professional has adjusted the hearing aid to meet your needs. If you have further questions, please contact your hearing care professional.

Intended use

The hearing aid is intended to amplify and transmit sound to the ear and thereby compensate for impaired hearing within mild to moderate to profound hearing loss. This power hearing aid is intended to be used by adults and children older than 36 months.

Important notice

The hearing aid amplification is uniquely adjusted and optimized to your hearing capabilities during the hearing aid fitting performed by your hearing care professional.

Table of contents

Identify your hearing aid and earpiece	10
Your hearing aid with earhook and mold – What it is	12
What it does	13
Your hearing aid with miniFit thin tube – What it is	14
What it does	15
Identify left and right hearing aid	16
Battery (size 13)	17
Turn the hearing aid ON and OFF	18
When to replace a battery	19
Replace the battery	20
Put on the hearing aid with earhook	22
MultiTool for handling batteries and cleaning	23
Caring for your hearing aid	24
miniFit thin tube	27
Standard earpieces for thin tube	30


Replace dome or Grip Tip	31
Customized earpieces for thin tube	32
Clean Micro Mold	33
Replace ProWax filter	34
Flight mode	35
Optional features and accessories	36
Change volume (optional)	37
Change program (optional)	38
Mute the hearing aid (optional)	40
Quick reset and pairing mode	41
Using hearing aid with iPhone, iPad, and iPod touch	42
Pairing hearing aid with iPhone	43
Re-connect your hearing aids to your iPhone, iPad, or iPod touch	46
Wireless accessories (optional)	47
Other options (optional)	48
Tamper-resistant battery drawer (optional)	49
Tinnitus SoundSupport (optional)	51

Guidelines for tinnitus sound generator users	52
Sound options and volume adjustment	53
Limitation on use time	55
Important information for hearing care professionals about Tinnitus SoundSupport	56
Warnings related to Tinnitus SoundSupport	58
General warnings	60
Troubleshooting guide	66
Water & dust resistance (IP68)	68
Conditions of use	70
Warranty certificate	71
International warranty	72
Technical information	73
Your individual hearing aid settings	75


Identify your hearing aid and earpiece

There are two different attachments and several different earpieces which classify your hearing aid. Please identify your attachment and earpiece. This will make it easier for you to navigate through this booklet.

Earhook with mold

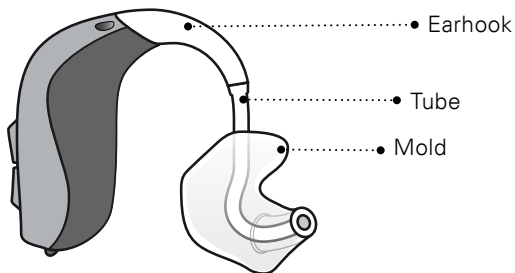

miniFit thin tube

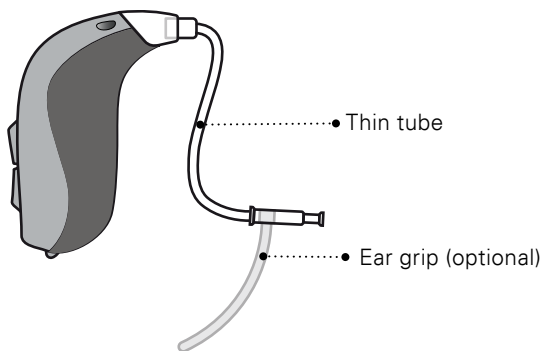

Earpieces

The thin tube uses one of the following earpieces:



Dome (Bass Dome shown)*



Grip Tip*



VarioTherm® Micro Mold




Micro Mold


* Please see details for replacing the dome or Grip Tip in section: "Replace dome or Grip Tip".

VarioTherm® is a registered trademark of Dreve.


Your hearing aid with earhook and mold – What it is

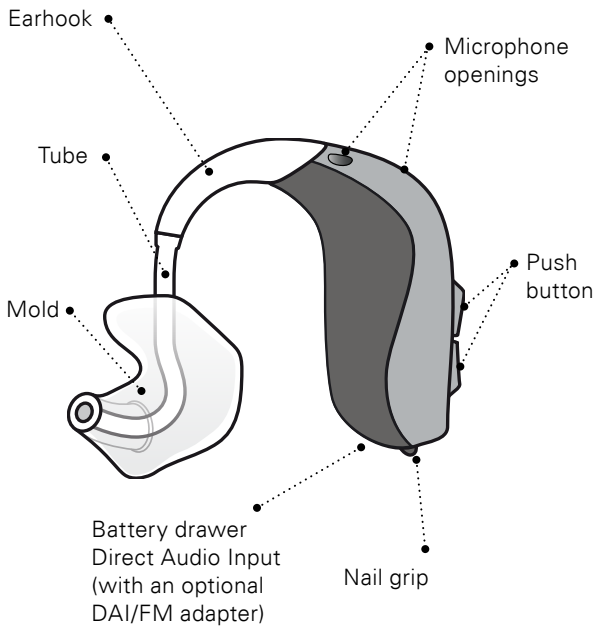


What it does

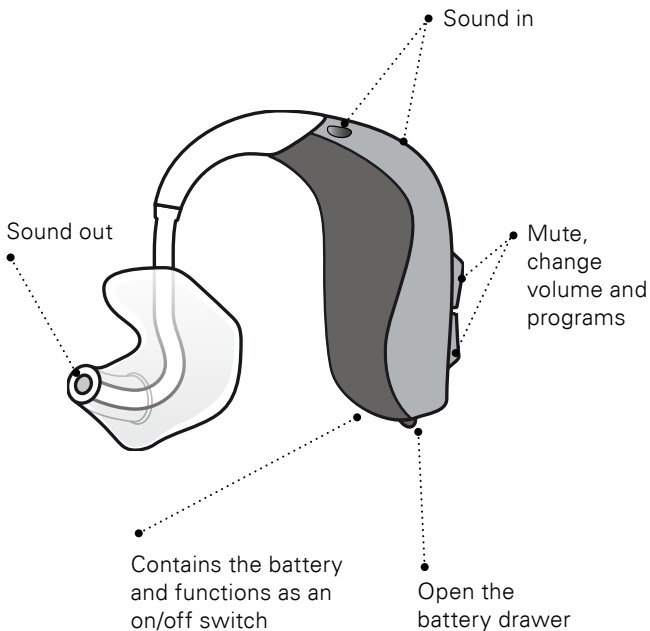

Your hearing aid with miniFit thin tube – What it is

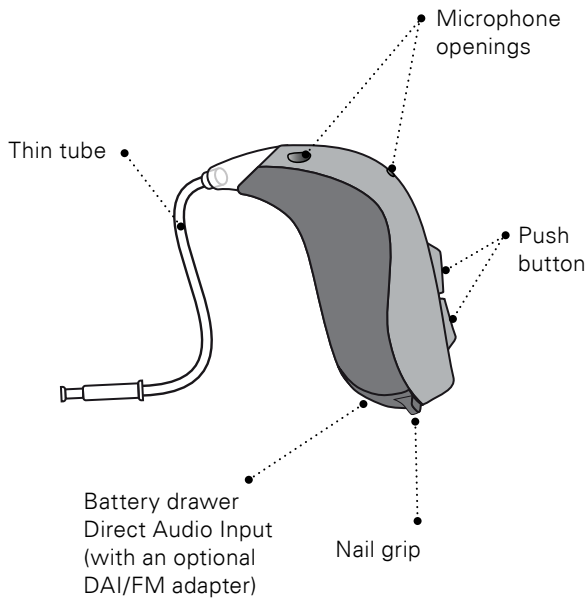


What it does

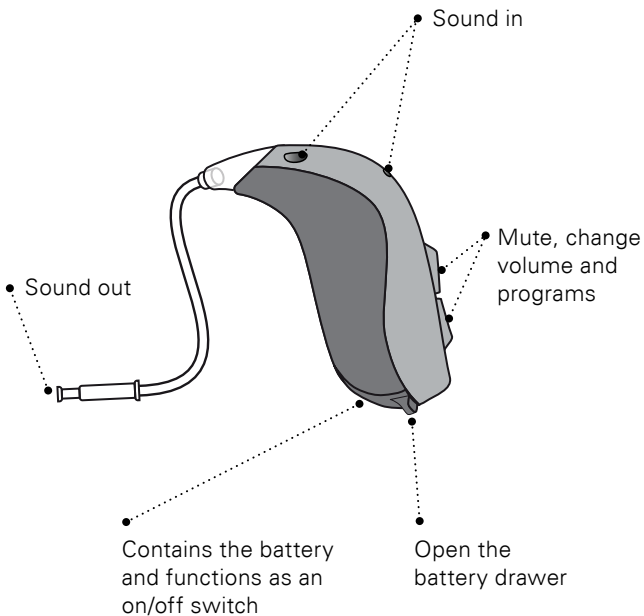

Identify left and right hearing aid

It is important to distinguish between the left and the right hearing aid as they might be programmed differently. You can find left/right color indicators in the battery drawer. Indicators can also be found on the molds.

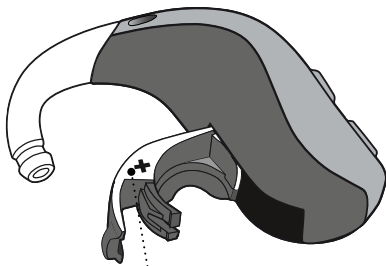

- A **RED** indicator marks the RIGHT hearing aid


A **BLUE** indicator marks the LEFT hearing aid

Battery (size 13)

Your hearing aid is a miniature electronic device that runs on special batteries. To activate the hearing aid, you must insert a new battery in the battery drawer. See how in the “Replace the battery” section.

Make sure the plus
is visible

Battery drawer

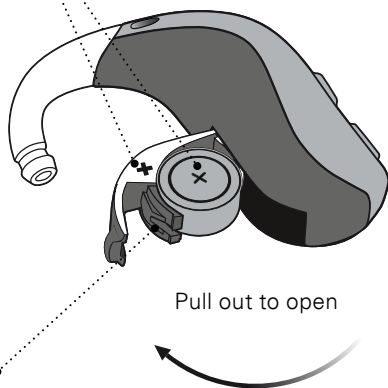


Pull out to open

Turn the hearing aid ON and OFF

The battery drawer is used to switch the hearing aid on and off. To save battery life, make sure your hearing aid is switched off when you are not wearing it.

Turn ON

Close the battery drawer with the battery in place.

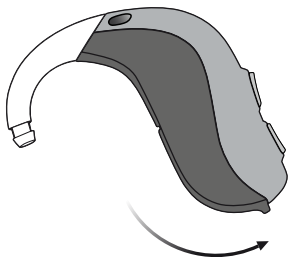

Turn OFF

Open the battery drawer.



When to replace a battery

When it is time to replace the battery, you will hear three beeps repeated at moderate intervals until the battery runs out.



Three beeps*

= The battery
is running low



Four beeps

= The battery
has run out

Battery tip

To make sure the hearing aid is always working, bring spare batteries with you, or replace the battery before you leave home.

* Bluetooth® will be shut down and is not possible to use wireless accessories.

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by William Demant Holding A/S is under license. Other trademarks and trade names are those of their respective owners.




The batteries need to be replaced more often if you are streaming audio or music to your hearing aid.

Replace the battery

1. Remove

Fully open the battery drawer. Remove the battery.

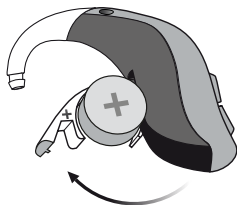


2. Uncover

Remove the sticky label from the + side of the new battery.


Tip:

Wait 2 minutes so that the battery can draw air, to ensure optimal function.

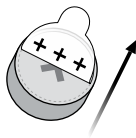

3. Insert

Insert the new battery into the battery drawer. Make sure the + side faces up.

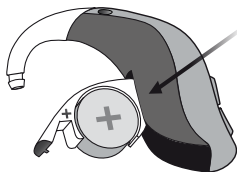


4. Close

Close the battery drawer. The hearing aid will play a jingle through the earpiece. Hold the earpiece close to your ear to hear the jingle.



Tip

The MultiTool can be used for battery change. Use the magnetic end to remove and insert batteries. The MultiTool is provided by your hearing care professional.

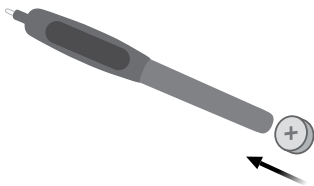

Put on the hearing aid with earhook

Read the following pages if you have a hearing aid with an earhook and mold.

The mold is customized for you and fits your ear shape. The molds are unique for the left or the right ear.

Step 1

Gently pull your ear outwards and press the mold in the direction of the ear canal, twisting slightly.




Step 2

Place the hearing aid behind your ear.



MultiTool for handling batteries and cleaning

The MultiTool contains a magnet which makes it easier to replace the battery in the hearing aid. It also contains a brush and a wire loop for cleaning and removing ear wax from your earpiece. If you need a new MultiTool, please contact your hearing care professional.

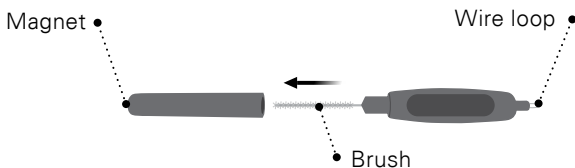

Important notice


The MultiTool has a built-in magnet. Keep the MultiTool at least 30 cm away from credit cards and other magnetically sensitive devices.

Caring for your hearing aid

When handling your hearing aid, hold it over a soft surface to avoid damage if you drop it.

Clean the microphone openings

Carefully brush away debris from the openings using a clean brush. Gently brush the surface and make sure the brush is not pressed into the openings.

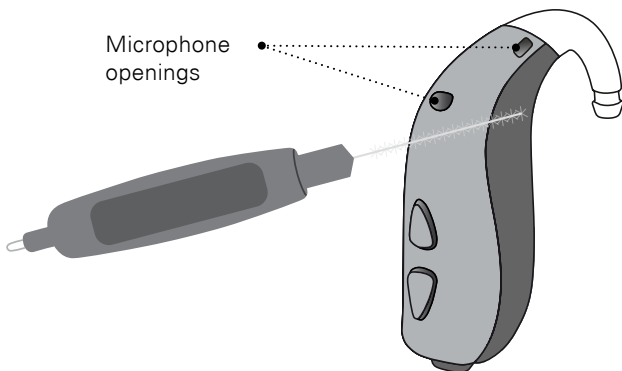


Important notice

Use a soft, dry cloth to clean the hearing aid. It must never be washed or immersed in water or other liquids.

Clean the mold

The mold should be cleaned regularly. Use a soft cloth to clean the surface of the mold. Use the MultiTool wire loop to clean the opening.

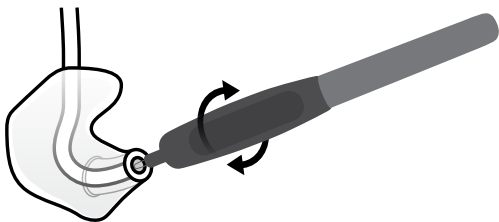

Replace the tube

The tubing between the mold and the hearing aid should be replaced when it becomes yellow or stiff. Consult your hearing care professional for tubing replacement.

Wash the mold


Step 1

Make sure you are able to identify which mold is for which hearing aid before disconnecting them from the hearing aid.




Step 2

Disconnect the tube and mold from the earhook. Hold the earhook firmly, when pulling off the tube. Otherwise the hearing aid may be damaged.

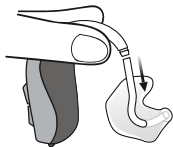

Step 3


Wash the mold and tube using mild soap. Rinse and dry completely before reconnecting to the hearing aid.

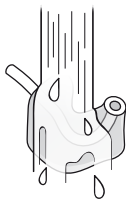

miniFit thin tube

Read the following pages if your hearing aid has a miniFit thin tube with either a dome, Grip Tip, or Micro Mold as earpiece.

miniFit is a thin tube that brings the sound into your ear. The thin tube should only be used with an earpiece attached. Use only parts designed for your hearing aid.

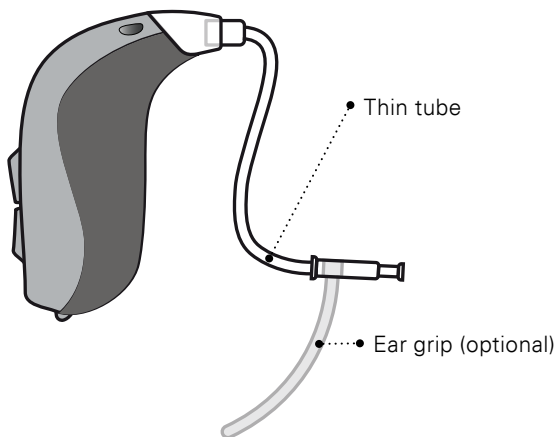

Put on the hearing aid with thin tube


Step 1

Place the hearing aid behind your ear.



Step 2

Hold the bend of the tube between your thumb and index finger. The earpiece should point towards the ear canal opening.

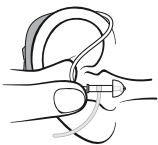

Step 3

Gently push the earpiece into your ear canal until the thin tube sits close against the side of your head. If the thin tube has an ear grip, place it in the ear so it follows the contour of the ear.




Clean the thin tube

The thin tube should be cleaned to keep it clear from moisture and wax. If not cleaned, it could become blocked with wax which means no sound will be able to come out.


Step 1

Pull the thin tube out of the hearing aid.

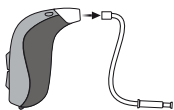


Step 2

Push the cleaning tool all the way through the thin tube.

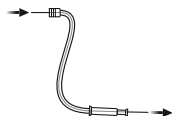

Step 3

Reconnect the thin tube to the hearing aid.

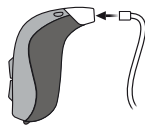

Standard earpieces for thin tube

There are 3 different types of domes. Both domes and Grip Tip are made of soft rubber material. Identify your earpiece type and size below.



Open Dome



Power Dome



Bass Dome, single vent



Bass Dome, double vent



Grip Tip (available in small and large, left and right, with or without vent)



6 mm



8 mm



10 mm



12 mm*

* not for Open Dome

Important notice


If the earpiece is not on the thin tube when removed from the ear, the earpiece might still be in the ear canal. Consult your hearing care professional for further instructions.

Replace dome or Grip Tip

The earpiece (dome or Grip Tip) should not be cleaned. If the earpiece is filled with wax, replace it with a new one. Grip Tip should be replaced at least once a month.


Step 1

Hold on to the end of the thin tube and then pull off the earpiece.

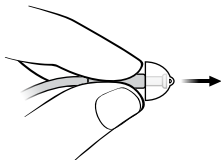


Step 2

Insert the thin tube exactly into the middle of the earpiece to obtain a firm attachment.

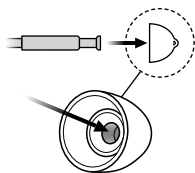


Step 3

Push firmly to ensure that the earpiece is fastened securely.

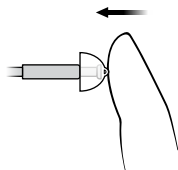

Customized earpieces for thin tube

Micro Molds are available in two different materials. The earpieces are customized for your ear.

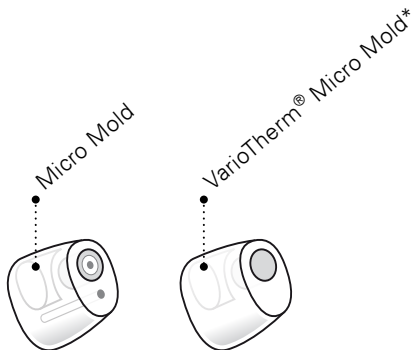


Customized earpieces have a white wax filter attached to the sound opening. ProWax is the filter type to be used with these earpieces. To replace the ProWax filter, see section “Replace ProWax filter”.

* VarioTherm® Micro Mold does not have a wax filter.

Clean Micro Mold

The Micro Mold should be cleaned regularly.

The Micro Mold has a white wax filter that must be replaced when clogged, or if the hearing aid does not sound normal.

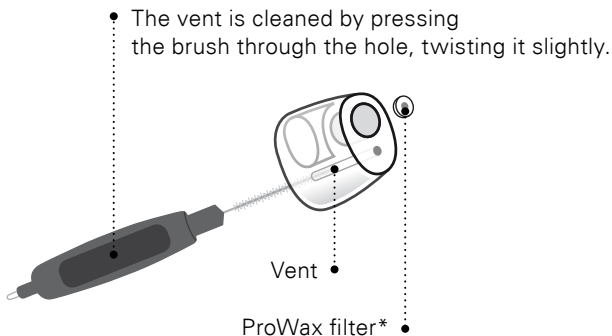

* VarioTherm® Micro Mold and Lite Tip do not have a wax filter.


Important notice

Always use the same type of wax filter as was originally supplied with the hearing aid. If you are in any doubt about the use or replacement of wax filters, contact your hearing care professional.

Replace ProWax filter


1. Tool

Remove the tool from the shell. The tool has two pins, one empty for removal and one with the new wax filter.

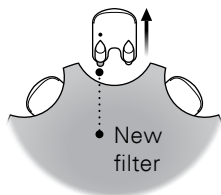


2. Remove

Push the empty pin into the wax filter in the mold and pull it out.

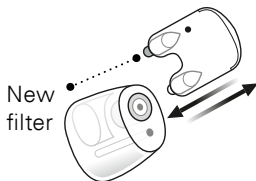


3. Insert

Insert the new wax filter using the other pin, remove the tool, and throw it out.

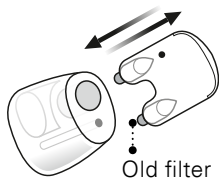

Flight mode

When boarding an airplane or entering an area where it is prohibited to radiate radio signals, e.g., during flight, flight mode must be activated. The hearing aid will still be working. It is only necessary to activate flight mode on one hearing aid.

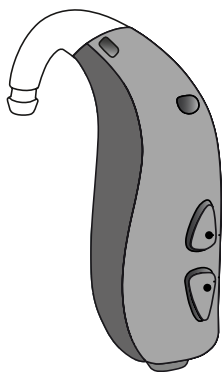

- **To activate and deactivate flight mode**

Press either end of the push button for at least 7 seconds. A jingle confirms your action.

Opening and closing the battery drawer will deactivate the flight mode. See section “Quick reset”.



Optional features and accessories

The features and accessories described on the following pages are optional. Please contact your hearing care professional to find out how your hearing aid is programmed. If you experience difficult listening situations, a special program may be helpful. These are programmed by your hearing care professional.

Write down hearing situations where you may need help.

Change volume (optional)

The push button allows you to adjust the volume. You will hear a click when you turn the volume up or down.

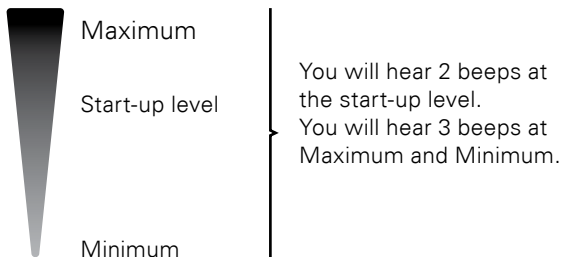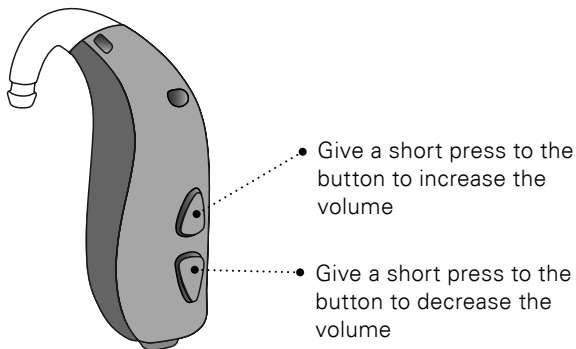


Volume Change

<input type="checkbox"/> Left	<input type="checkbox"/> Right	
-------------------------------	--------------------------------	--

To be filled out by the hearing care professional.

□ Change program (optional)



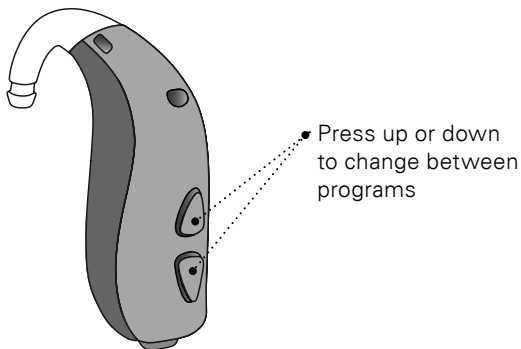

Your hearing aid can have up to 4 different programs. These are programmed by your hearing care professional.



*

You can change continuously between programs both up and down in the program order. For example, if you want to go from program 1 to 4, you can press the down button once instead of pressing the up button 3 times.

To be filled out by the hearing care professional


Program	Activation sound	When to use
1	 "1 beep"	
2	 "2 beeps"	
3	 "3 beeps"	
4	 "4 beeps"	

Program change:

<input type="checkbox"/> Left	<input type="checkbox"/> Right	<input type="checkbox"/> Short press	<input type="checkbox"/> Medium press
-------------------------------	--------------------------------	--------------------------------------	---------------------------------------

□ Mute the hearing aid (optional)

Use the mute function if you need to silence the hearing aid while wearing it.

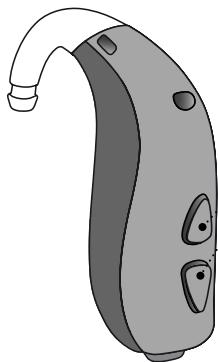

Apply a very long press (4 seconds) to either end of the button to mute the hearing aid.

To reactivate, push the button briefly.

Your hearing aid can be muted by using one of the following optional devices:

- Bernafon EasyControl-A app
- SoundClip-A
- RC-A remote control

Important notice

Do not use the mute function as an off switch, as the hearing aid still draws current from the battery in this mode.

Quick reset and pairing mode


If you wish to return to the standard settings of the hearing aid programmed by your hearing care professional, simply open and then close the battery drawer. A jingle is played when you reset the hearing aid.

To pair your hearing aid with a wireless accessory, your hearing aid must be put into pairing mode. This can be done by opening and closing the battery drawer. The hearing aid will remain in pairing mode for 180 seconds. All compatible accessories are listed in chapter “Wireless accessories”.

Open



Close

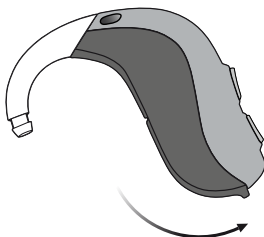

Using hearing aid with iPhone, iPad, and iPod touch

Your hearing aid is a Made for iPhone® hearing aid and allows for direct communication and control with an iPhone®, iPad® or iPod® touch.

For assistance in pairing and using these products with your hearing aid, please contact your hearing care professional or visit our support site at: www.bernafon.com/library.


For information on compatibility, please visit www.bernafon.com/products/accessories.



Direct audio streaming is currently not supported by Android™.


Pairing hearing aid with iPhone

1. Settings

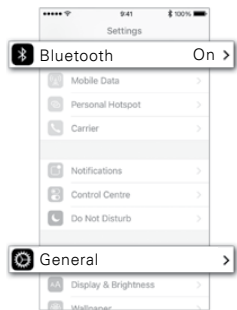


Open your iPhone and go to "Settings". Make sure Bluetooth® is on. Then choose "General".

2. General

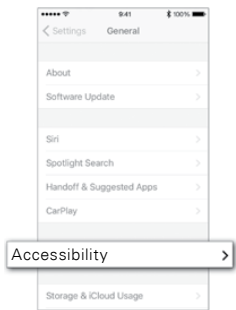


On the "General" screen, choose "Accessibility".

3. Accessibility

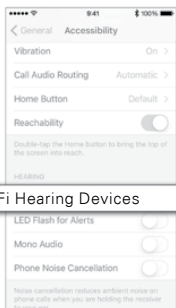


On the “Accessibility” screen, choose “MFi Hearing Devices”.

4. Prepare

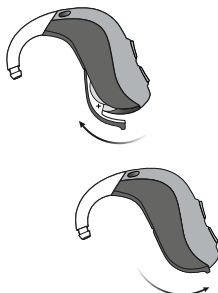


Open and close the battery drawer, on both hearing aids if you have two and place it close to your iPhone. The hearing aid stays in pairing mode for 3 minutes.

5. Select

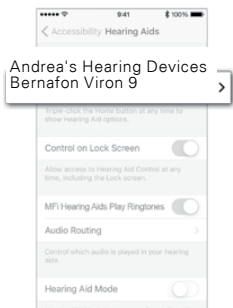

Your iPhone will detect the hearing aid for pairing. Detected devices will appear in your iPhone list. Choose your hearing aid.

6. Confirm pairing

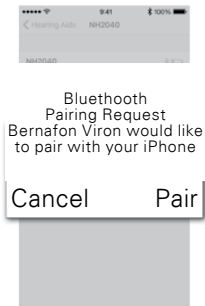

Confirm pairing. If you have two hearing aids, pairing confirmation is needed for each hearing aid.

Re-connect your hearing aids to your iPhone, iPad, or iPod touch

When you turn off your hearing aids, iPhone, iPad, or iPod touch, these will no longer be connected. To connect them again, turn on your hearing aids by opening and closing the battery drawer. The hearing aids will then automatically re-connect to your device.

Use of the Made for Apple badge means that an accessory has been designed to connect specifically to the Apple product(s) identified in the badge, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPod, iPhone, or iPad may affect wireless performance.

Wireless accessories (optional)

As an enhancement to your wireless hearing aid, a range of wireless accessories are available. They can enable you to hear and communicate better in a lot of everyday situations.

SoundClip-A

When this device is paired with your mobile phone, you can use the hearing aids like stereo headphones. SoundClip-A can also function as a remote microphone and remote control.

TV-A

TV-A adapter is a wireless transmitter of sound from TV and electronic audio devices. The TV-A streams sound directly to your hearing aid.

RC-A

RC-A remote control offers the ability to change programs, adjust volume or mute your hearing aid.

Bernafon EasyControl-A app

The app offers an intuitive and discreet way to control your hearing aids with iPhone, iPad, iPod touch, and Android devices.

For more information, please contact your hearing care professional or visit:
www.bernafon.com/products/accessories.

Other options (optional)

Telecoil

Telecoil helps you hear better when using a telephone with a built-in loop or when you are in buildings with teleloop systems such as theatres, churches, or lecture rooms. This symbol or a similar sign is shown wherever a teleloop has been installed.



Auto Telephone (optional for Zerena)

The Auto Telephone can automatically activate a phone program in the hearing aid, if your telephone has a dedicated magnet. The magnet needs to be placed on your telephone next to the sound outlet.



Direct Audio Input (DAI) adapter

Allows your hearing aid to receive signals directly from external sources such as TV, personal stereos, etc.

FM adapter

Allows your hearing aid to connect to an external FM receiver.

For more information, please contact your hearing care professional.

Tamper-resistant battery drawer (optional)

To keep the battery out of reach of infants, small children and people with learning difficulties, a tamper-resistant battery drawer should be used.

Important notice

Avoid excessive force to open the battery drawer when in the locked position. Do not force the battery drawer beyond its fully opened position. Make sure to insert the battery correctly. Please contact your hearing care professional if you suspect the tamper-resistant effectiveness and locking ability are damaged.


To lock the battery drawer

Close the battery drawer completely and make sure it is locked.



Tool to unlock the battery drawer


Use this small tool provided by your hearing care professional.

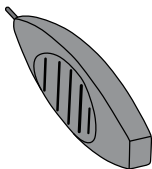

Unlock the battery drawer

Insert the tip of the tool into the small hole at the back of the battery drawer.

1. Move it to the right and hold it.
2. Open the battery drawer.

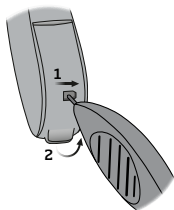

□ Tinnitus SoundSupport (optional)

Intended use of Tinnitus SoundSupport

Tinnitus SoundSupport is a tool intended to generate sounds to provide temporary relief for patients suffering from tinnitus as part of a tinnitus management program.

The target population is the adult population (over 18 years old).

Tinnitus SoundSupport is targeted to licensed hearing care professionals (audiologists, hearing aid specialists, or otolaryngologists) who are familiar with the evaluation and treatment of tinnitus and hearing loss. Fitting of Tinnitus SoundSupport must be done by a hearing care professional participating in a tinnitus management program.

Guidelines for tinnitus sound generator users

These instructions contain information about Tinnitus SoundSupport, which may have been enabled in your hearing aids by your hearing care professional.

Tinnitus SoundSupport is a tinnitus management device intended to generate sound of sufficient intensity and bandwidth to help manage tinnitus.

Your hearing care professional will also be able to offer the appropriate follow-up care. It is important to follow his/her advice and directions regarding such care.

Prescription use only

Good health practice requires that a person reporting tinnitus has had a medical evaluation by a licensed ear physician before using a sound generator. The purpose of such an evaluation is to ensure that any medically treatable condition that may cause tinnitus is identified and treated prior to using a sound generator.

Sound options and volume adjustment

Tinnitus SoundSupport is programmed by your hearing care professional to match your hearing loss and preferences for tinnitus relief. It offers a number of different sound options. Together with your hearing care professional, you can select your preferred sound(s).

Tinnitus SoundSupport programs

Together with your hearing care professional, you decide for which programs you may want to have Tinnitus SoundSupport activated. The sound generator can be activated in up to four different programs.

Mute

If you are in a program for which Tinnitus SoundSupport is activated, the mute functionality will mute only the environmental sounds, and not the sound from Tinnitus SoundSupport. See chapter: "Mute the hearing aid".

Volume adjustments with Tinnitus SoundSupport

When you select a hearing aid program for which Tinnitus SoundSupport is activated, your hearing care professional can only set the push button on your hearing aid to work as a volume control for the tinnitus relief sound.

Your hearing care professional will set the volume control for the sound generator in one of two ways:

- A) change volume in each ear separately, or
- B) change volume in both ears simultaneously.

A) How to change Tinnitus SoundSupport volume in each ear separately

To increase volume (on one hearing aid only), use a short press on the upper part of the push button repeatedly until desired level is reached.

To decrease volume (on only one hearing aid), use a short press on the lower part of the push button repeatedly until desired level is reached.

B) How to change Tinnitus SoundSupport volume in both ears simultaneously

You can use one hearing aid to increase/decrease the sound in both hearing aids. When changing the volume in one hearing aid, the volume on the other hearing aid will follow. To increase volume, use a short press on the upper part of the push button repeatedly. To decrease volume, use a short press on the lower part of the push button repeatedly.

To be filled out by your hearing care professional.

Limitation on use time

Daily use

The volume levels of Tinnitus SoundSupport can be set to a level which could lead to permanent hearing damage when used for a prolonged period of time. Your hearing care professional will advise you of the maximum amount of time per day you should use Tinnitus SoundSupport. It should never be used at uncomfortable levels.

See table “Tinnitus SoundSupport: Limitation on use” at the end of this booklet to learn how many hours per day you can safely use the relief sound in your hearing aids.

Important information for hearing care professionals about Tinnitus SoundSupport

Device description

Tinnitus SoundSupport is a module function that can be enabled in the hearing aids by the hearing care professional.

Maximum wearing time

The wearing time of Tinnitus SoundSupport will decrease as you increase the level above 80 dB(A) SPL. The fitting software will automatically display a warning when the hearing aid exceeds 80 dB(A) SPL. See “max wearing time” indicator next to the tinnitus fitting graph in the fitting software.

The volume control is deactivated

By default, the volume control for the sound generator is deactivated in the hearing aid. Risk of noise exposure increases when the volume control is activated.

If the volume control is activated

A warning may be displayed if you activate the tinnitus volume control in the “Local Controls” screen. This occurs if the relief sound can be listened to at levels that may cause hearing damage. The “Limitation on Use” table in the fitting software displays the number of hours the patient can safely use Tinnitus SoundSupport.

- Note the max wearing time for each program for which Tinnitus SoundSupport is activated.
- Write those values in the table: “Tinnitus SoundSupport: Limitation on use”, in the back of this booklet.
- Instruct your patient accordingly.

The Tinnitus SoundSupport is manufactured for Bernafon AG.

⚠ Warnings related to Tinnitus SoundSupport

If your hearing care professional has activated the sound generator Tinnitus SoundSupport, please pay attention to the following warnings.

There are some potential concerns associated with the use of any sound generated by a tinnitus management device. Among them are the potential worsening of tinnitus, and/or a possible change in hearing thresholds.

Should you experience or notice a change in hearing or tinnitus, or any dizziness, nausea, headaches, heart palpitations, or possible skin irritation at the point of contact with the device, you should immediately discontinue use of the device and consult a medical, audiology, or other hearing care professional.

As with any device, misuse of the sound generator feature may cause potentially harmful effects. Care should be taken to prevent unauthorized use and to keep the device out of reach of children and pets.

Maximum wearing time

Always follow the maximum wearing time per day of the Tinnitus SoundSupport advised by your hearing care professional. Prolonged use may lead to worsening of your tinnitus or of your hearing loss.

⚠ General warnings

You should familiarize yourself fully with the following general warnings before using your hearing aid for your personal safety and to ensure correct use.

Please note that a hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Furthermore, note that in most cases, infrequent use of a hearing aid does not permit a user to attain full benefit from it.

Consult your hearing care professional if you experience unexpected operations or events with your hearing aid.

Usage of hearing aids

- Hearing aids should be used only as directed and adjusted by your hearing care professional. Misuse can result in sudden and permanent hearing loss.
- Never allow others to wear your hearing aid, as incorrect usage could cause permanent damage to their hearing.

Choking hazards & risk of swallowing batteries and other small parts

- Hearing aids, their parts, and batteries should be kept out of reach of children and anyone who might swallow these items or otherwise cause injury to themselves.
- Batteries have occasionally been mistaken for pills. Therefore, check your medicine carefully before swallowing any pills.
- Most hearing aids can be supplied with a tamper-resistant battery drawer upon request. This is strongly recommended for infants, small children, and people with learning difficulties.
- Children younger than 36 months must always use a tamper-resistant battery drawer. Please talk to your hearing care professional about the availability of this option.

If a battery, hearing aid or small part is swallowed, see a doctor immediately.

Battery use

- Always use batteries recommended by your hearing care professional. Batteries of low quality may leak and cause bodily harm.
- Never attempt to recharge your batteries, and never dispose of batteries by burning them. There is a risk that the batteries will explode.

General warnings

Dysfunction

- Be aware of the possibility that your hearing aid may stop working without notice. Keep this in mind when you depend on warning sounds (e.g. when you are in traffic). The hearing aids may stop functioning, for instance if the batteries have expired or if the tubing is blocked by moisture or ear wax.

Active implants

- The hearing aid has been thoroughly tested and characterized for human health according to international standards for human exposure (Specific Absorption Ratio – SAR), induced electromagnetic power and voltages into the human body. The exposure values are well below international accepted safety limits for SAR, induced electromagnetic power and voltages into the human body defined in the standards for human health and coexistence with active medical implants as pacemakers and heart defibrillators.
- The Auto Telephone magnet and the MultiTool (which has a built-in magnet) should be kept more than 30 cm away from the implant, e.g. do not carry it in a breast pocket.
- If you have an active brain implant, please contact the manufacturer of your implantable device for information about the risk of disturbance.

X-ray, CT, MR, PET scanning and electrotherapy

- Remove your hearing aid before X-ray, CT/MR/ PET scanning electrotherapy, surgery etc. as your hearing aid may be damaged when exposed to strong electromagnetic fields.

Heat and chemicals

- The hearing aid must never be exposed to extreme heat e.g. left inside a parked car in the sun.
- The hearing aid must not be dried in microwave ovens or other ovens.
- The chemicals in cosmetics, hairspray, perfume, aftershave lotion, suntan lotion and insect repellent can damage the hearing aid. Always remove your hearing aid before applying such products and allow time to dry before use.

Power instrument

- Special care should be exercised in selecting, fitting and using a hearing aid where maximum sound pressure capability exceeds 132 dB SPL (IEC 711), as there may be risk of impairing the remaining hearing of the hearing aid user.

General warnings

Possible side effects

- Hearing aids, molds or domes may cause an accelerated accumulation of earwax.
- The otherwise non-allergenic materials used in hearing aids may in rare cases cause a skin irritation or other side effects.

Please seek consultation with a physician if these conditions occur.

Interference

- The hearing aid has been thoroughly tested for interference, according to the most stringent international standards. However, interference between the hearing aid and other devices (e.g. some mobile telephones, citizens band systems and shop alarm systems and other devices) may occur. If this occurs, increase the distance between the hearing aid and the interfering device.

Use on aircraft

- Your hearing aid contains Bluetooth®. On board an aircraft, flight mode must be activated or the hearing aid needs to be turned off, unless Bluetooth® is specifically permitted by the flight personnel.

Connection to external equipment

- The safety of the hearing aid when connected to external equipment with an auxiliary input cable and/or with a cable and/or directly, is determined by the external signal source. When connected to external equipment plugged into a wall outlet, this equipment must comply with IEC-62368 (or IEC-60065, IEC-60950 until June 20, 2019) or equivalent safety standards.

Important notice

When the DAI is connected to an audio source plugged into a wall outlet, the audio source must comply with EC-62368 (or IEC-60065, IEC-60950 until June 20 2019) or equivalent safety standards.

Troubleshooting guide

Symptom	Possible causes
No sound	Dead battery
	Clogged earpieces (dome, Grip Tip, or Micro Mold)
Intermittent or reduced sound	Clogged sound outlet
	Moisture
	Dead battery
Squealing noise	Hearing aid earpiece inserted incorrectly
	Ear wax accumulated in ear canal
Beeping	If your hearing aid plays 8 beeps, 4 times consecutively, your hearing aid needs a microphone service check
Pairing issue with Apple device	Bluetooth® connection failed
	Only one hearing aid paired

Solutions

Replace the battery

Clean mold

Consider replacing wax filter, dome, or Grip Tip

Clean mold or replace wax filter, dome, or Grip Tip

Wipe battery with a dry cloth

Replace the battery

Re-insert the earpiece

Have ear canal examined by your doctor

Contact your hearing care professional

1. Unpair your hearing aids
(Settings → General → Accessibility → MFi Hearing Devices → Devices → Forget this device).
 2. Turn Bluetooth® off and on again.
 3. Open and close battery drawer on hearing aids.
 4. Re-pair hearing aids
(see section "Pair hearing aid with iPhone").
-

If none of the above solutions work, consult your hearing care professional for assistance.

Water & dust resistance (IP68)

Your hearing aid is dust-tight and protected against ingress of water which means it is designed to be worn in all daily life situations. Therefore, you do not have to worry about sweat or getting wet in the rain. Should your hearing aid come in contact with water and stop working, please follow these guidelines:

1. Gently wipe off any water.
2. Open the battery drawer and remove the battery and gently wipe off any water in the battery drawer.
3. Let the hearing aid dry with the battery drawer left open for approximately 30 minutes.
4. Insert a new battery.

Important notice

Do not wear your hearing aid while showering or participating in water activities. Do not immerse your hearing aid in water or other liquids.

Conditions of use

Operating conditions	Temperature: +1 °C to +40 °C Relative humidity: 5 % to 93 %, non-condensing
----------------------	--

Storage and transportation conditions	Temperature and humidity should not exceed the following limits for extended periods during transportation and storage: Temperature: -25 °C to +60 °C Relative humidity: 5 % to 93 %, non-condensing
---------------------------------------	--

Warranty certificate

Name of owner:

Hearing care professional:

Hearing care professional address:

Hearing care professional phone:

Purchase date:

Warranty period:

Month:

Model right:

Serial no.:

Model left:

Serial no.:

International warranty

Your hearing aid is covered by an international limited warranty issued by the manufacturer for a period of 12 months from the date of delivery. This limited warranty covers manufacturing and material defects in the hearing aid itself, but not in accessories such as batteries, tubing, speakers, earpieces and filters, etc. Problems arising from improper/incorrect handling or care, excessive use, accidents, repairs made by an unauthorized party, exposure to corrosive conditions, physical changes in your ear, damage due to foreign objects entering the device or incorrect adjustments are NOT covered by the limited warranty and may void it. The above warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods. Your hearing care professional may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult him/her for further information.

If you need service

Take your hearing aid to your hearing care professional, who may be able to sort out minor problems and adjustments immediately.

Technical information

The hearing aid contains two radio technologies, which are described below:

The hearing aid contains a radio transceiver using short range magnetic induction technology working at 3.84 MHz. The magnetic field strength of the transmitter is very weak and is always below 15 nW (-40 dB μ A/m at 10 meter distance).

The hearing aid also contains a radio transceiver using Bluetooth® Low Energy (BLE) and a proprietary shortrange radio technology, both working at 2.4 GHz. The radio transmitter is weak and is always below 3 mW and equal to 4.8 dBm in total radiated power.

The hearing aid complies with international standards concerning electromagnetic compatibility and human exposure.

Due to the limited space available on the hearing aid, relevant approval markings can be found in this document.

Additional information can be found in the Product Information Sheet on www.bernafon.com.

USA and Canada

The hearing aid contains a radio module with the following certification ID numbers:

FCC ID: U6XAUBTEP

IC: 7031A-AUBTEP

The manufacturer declares that this hearing aid is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

Declaration of Conformity is available from the manufacturer.

Bernafon AG
Morgenstrasse 131
3018 Bern
Switzerland

CE 0543



Waste from electronic equipment must be handled according to local regulations.

Bluetooth®



IP68

Your individual hearing aid settings

To be filled out by your hearing care professional

Settings overview for your hearing aid

Left		Right
<input type="checkbox"/>	Volume control	<input type="checkbox"/>
<input type="checkbox"/>	Program shift	<input type="checkbox"/>
<input type="checkbox"/>	Mute	<input type="checkbox"/>
<input type="checkbox"/>	Tinnitus SoundSupport	<input type="checkbox"/>

Volume control indicators






<input type="checkbox"/>	Beeps at min/max volume	<input type="checkbox"/>
<input type="checkbox"/>	Clicks when changing volume	<input type="checkbox"/>
<input type="checkbox"/>	Beeps at preferred volume	<input type="checkbox"/>

Battery indicators

<input type="checkbox"/>	Low battery warning	<input type="checkbox"/>
--------------------------	---------------------	--------------------------

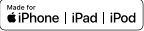Description of symbols used in this booklet or on the packaging label

	<p>Warnings Text marked with a marking symbol must be read before using the product.</p>
	<p>Manufacturer The product is produced by the manufacturer whose name and address are stated next to the symbol. Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.</p>
	<p>CE mark The product complies with Medical Device Directive 93/42/EEC. The four digits number indicates the identification of the notified body.</p>
	<p>Electronic waste (WEEE) Recycle your hearing aids, accessories or batteries according to local regulations or return them to your hearing care professional for disposal. Electronic equipment covered by Directive 2012/19/EU on waste and electrical equipment (WEEE).</p>
	<p>Regulatory Compliance Mark (RCM) The product complies with electrical safety, EMC and radio spectrum requirements for products supplied to the Australian or New Zealand market.</p>
	<p>IP code This symbol indicates the class of protections against harmful ingress of water and particulate matter according to EN 60529:1991/A1:2002. IP6X indicates total dust protection. IPX8 indicates the protection against the effects of continuous immersion in water.</p>
	<p>Bluetooth® logo Registered trademark of Bluetooth® SIG, Inc. where any use of such requires a license.</p>

	<p>Made for iPod, iPhone, and iPad Indicates that the device is compatible with iPod, iPhone and iPad.</p>
	<p>Keep dry Indicates a medical device that needs to be protected from moisture.</p>
	<p>Caution symbol Consult instructions for use for warnings and cautions.</p>
	<p>Catalogue number Indicates the manufacturer's catalogue number so that the medical device can be identified.</p>
	<p>Serial number Indicates the manufacturer's serial number so that a specific medical device can be identified.</p>

Tinnitus SoundSupport: Limitation on use

No limitation on use

Program	Start-up volume (Tinnitus)	Max volume (Tinnitus)
<input type="checkbox"/> 1	Max ____ hours per day	Max ____ hours per day
<input type="checkbox"/> 2	Max ____ hours per day	Max ____ hours per day
<input type="checkbox"/> 3	Max ____ hours per day	Max ____ hours per day
<input type="checkbox"/> 4	Max ____ hours per day	Max ____ hours per day

< >
8,15 mm

Placeholder
for FSC logo

The logo features a green circular emblem with a stylized tree in the center. Below the emblem, the text 'FSC' is written in a large, bold, serif font, followed by 'MIX' and 'Paper' in a smaller, sans-serif font. At the bottom of the logo, the FSC certification code 'FSC® C043870' is visible.

FSC
MIX
Paper
FSC® C043870

^
19 mm
v

^
9,0 mm
v

< 15 mm >

Australia

Bernafon Australia
629 Nudgee Road
Nundah QLD 4012
Freecall 1800 809 111
Phone +61 7 3250 0300
info@bernafon.com.au

United Kingdom

Bernafon UK
Cadzow Industrial Estate
Off Low Waters Road
Hamilton
ML3 7QE Scotland
Phone +44 1698 285 968
info@bernafon.co.uk



Manufacturer:

Bernafon AG
Morgenstrasse 131
3018 Bern
Switzerland
www.bernafon.com

New Zealand

Bernafon New Zealand
Millennium Centre
Level 2, Building A
600 Great South Road
Greenlane, Auckland 1051
Toll Free 0800 442 257
info@bernafon.co.nz

South Africa

Bernafon South Africa (Pty) Ltd
39 Van Vuuren Street
Constantia Kloof
1709
Phone +27 11 675 6104



0000200024000001

bernafon 
Your hearing • Our passion