

TO USE YOUR PHONE ADAPTER 2.0 WITH YOUR SOUNDCLIP-A

INCOMING CALLS (SC-A)

HOW TO RECEIVE A CALL

Make sure that you are able to speak into the SoundClip-A microphone.

When your landline phone rings, you will hear an incoming call alert in your hearing aids. The phone indicator on the phone adapter will blink green and at the same time the status indicator on the SoundClip-A will blink green.

Press the multibutton on the SoundClip-A briefly to answer the call. You will hear a small confirmation beep in your hearing aids. The status indicator on the SoundClip-A and the phone indicator on the phone adapter will now be steady green.

To end the call, simply press the multibutton on the SoundClip-A briefly and the status indicator turns blue.

IMPORTANT NOTICE

The SoundClip-A can only be paired to one device at a time.

OUTGOING CALLS (SC-A)

HOW TO MAKE A CALL

Make sure that you are able to speak into the SoundClip-A microphone.

1. To make a call press the SoundClip-A multibutton briefly. The hook is now off and you will hear a dial tone in the hearing aids. The status indicator on the SoundClip-A and the phone indicator on the phone adapter will turn steady green.
2. Pick up the handset on your landline phone (or press "Call" on the portable phone) and dial the phone number as you normally do to make a call.
3. After a short while you will hear the call tone in your hearing aids indicating that the call is now connected through your SoundClip-A and hearing aids.

You can now put the handset back on the phone (or press "end call" on the portable phone). To end the call, simply press the multibutton on the SoundClip-A briefly.

*Please make sure to put the handset correctly back on the phone otherwise the phone line will block for Incoming calls.