


Quick guide

Battery replacement / Bernafon miniRITE T R




Tools needed: Service tool (red), part number: 890-22-2270-00


Replacement of the Li-ion battery can only be performed by trained hearing care professionals.

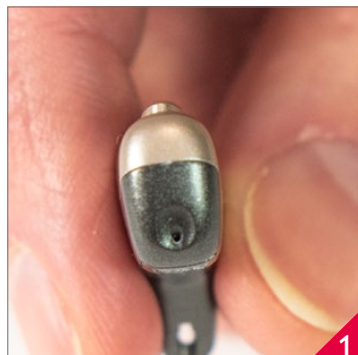

Look for the battery drawer release hole on the bottom of the instrument.

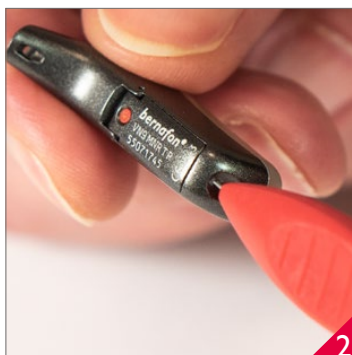

Insert the service tool at an angle as shown in the battery drawer release hole to open the battery drawer.

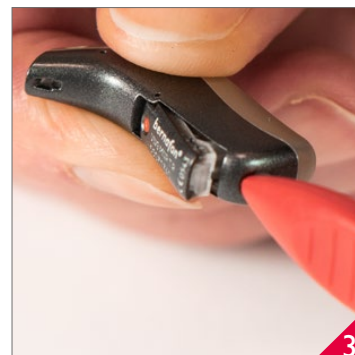

The battery drawer has been released.



Open battery drawer fully.




Remove the Li-ion battery and recycle it according to the country specifications.



Take a new battery. Use gloves to avoid moisture or oil from coming into contact with the new battery.



Use gloves to place the new battery as shown with the + sign facing up for both the battery and the battery drawer.

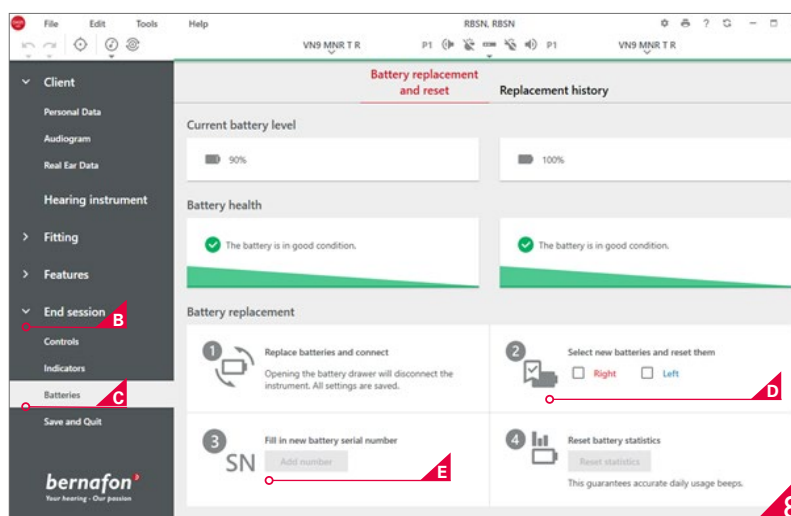


Reset the battery statistics in Oasis^{next}.

- A. Connect the instruments
- B. Click **End Session**
- C. Click **Batteries**
- D. Click **Right** or **Left** or both
- E. Click **Add number**


Battery replacement / Bernafon miniRITE T R

F. Enter the 10-digit serial number of the Li-ion battery


G. Click **Save**

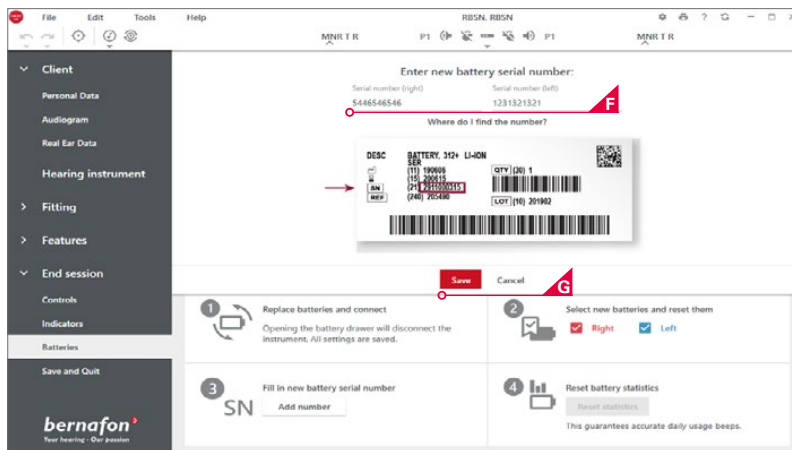

H. Click **Reset statistics**

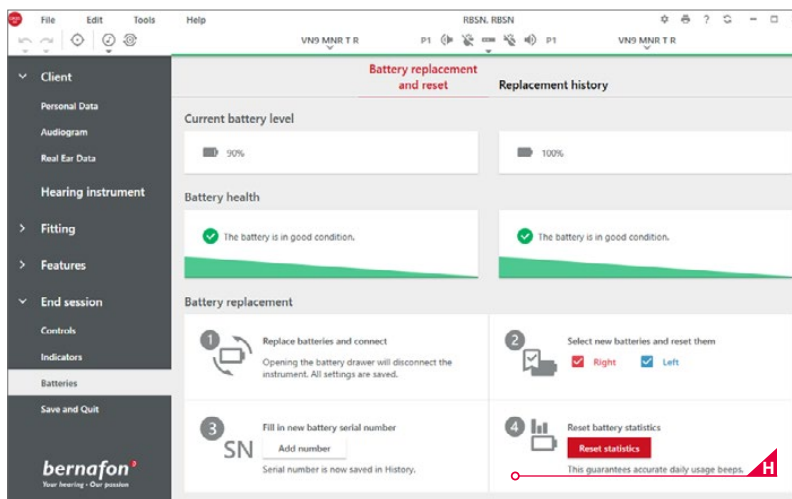

I. View details of the battery replacement in **Replacement history**

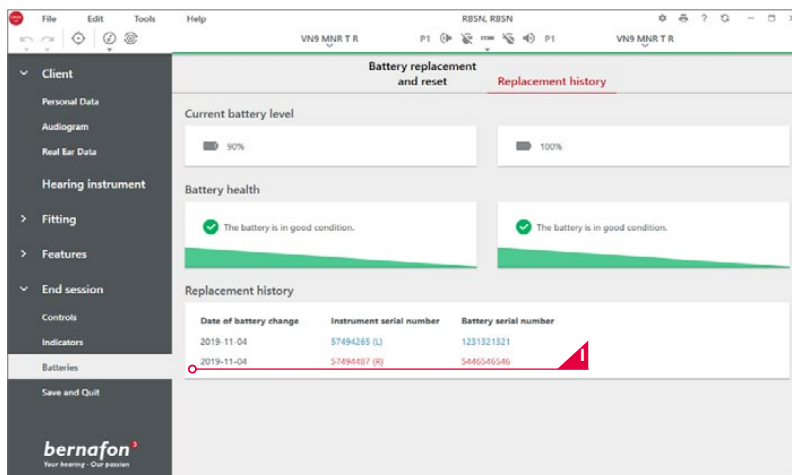

J. Advise the client to fully charge the new battery at home to ensure a full day's performance.