

Programming cables and adapters / Programmierkabel und -adapter Câbles et adaptateurs de programmation / Cables y adaptatores para programación

Use the **FittingLink** wireless programmer for the wireless hearing instruments (remember to insert a battery).
Otherwise use the cables below. For the families and styles listed below use cable **HI-PRO #2** (2 meter).


HI-PRO #2 **Alternative NOAHlink #2** (50 cm)
Left: 384-20-033-00 Left: 384-20-035-00
Right: 384-20-032-00 Right: 384-20-034-00




Family	Styles	
Juna 9 7 Saphira 5 3 Nevara 1	CIC*/IIC*	A/B
	ITE/ITC/IICx	B
	Pico RITE	C
	CPx/CP BTE	D
	Power BTE/Nano BTE/Nano RITE	E
Supremia 7 3	SP BTE	D
Acrida 9 7 Carista 5 3	ITE/ITC/CIC/IIC	B
	CPx/CP BTE	D
	Nano BTE/Nano RITE	E
Chronos 9 7 5 Inizia 3 1	CPx/CP BTE	D
	Nano BTE/Nano RITE	E
	ITED/ITC/CIC	C
	ITCD	F

* CIC and IIC instruments can come with two different faceplates.
Please be aware that a specific faceplate will only fit with the corresponding programming adapter.


Note: Always insert a fresh battery prior to programming.

 Exception: **Power BTE** (see instruction **E**)


A Programming adapter mini




164237

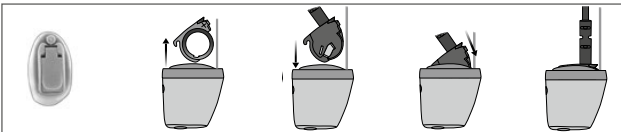


B FlexConnect Mini




117468

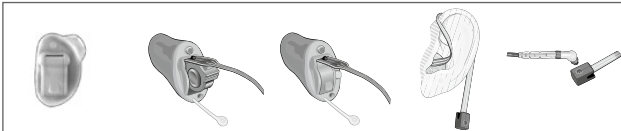


C FlexConnect

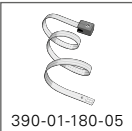


390-01-180-05

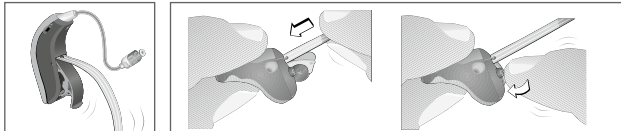

D Programming adapter




399-50-640-00

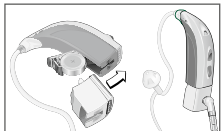


E Direct connection


Power BTE




Nano BTE



Nano RITE



F Programming strip



390-01-040-00

