


## Pairing your ConnectClip with your instrument(s)

---

1. Ensure ConnectClip is powered off.

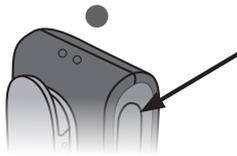


Press and hold the multibutton for approx. 6 seconds until power indicator OFF ●

2. Turn instrument(s) OFF.



(open battery drawer)

3. Turn instrument(s) ON.

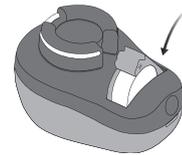

(close battery drawer)

4. The instrument(s) will now be ready for a first time connection with the ConnectClip for about 3 minutes

5. Turn ON ConnectClip.




Press and hold the multibutton for approx. 3 sec. until power indicator STEADY Green ●

6. ConnectClip will now search for your instrument(s).



Status indicator BLINKING YELLOW ●

7. Place ConnectClip and instrument(s) close to each other

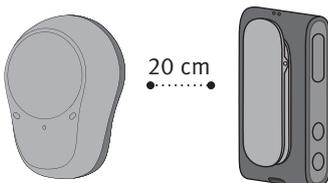

(max distance 20 cm)

8. Connection established with instrument(s)



is indicated by status indicator STEADY YELLOW ●


## Pairing your iPhone with your instrument(s)

### 1. Settings




Go to your iPhone settings. Check to make sure that Bluetooth is on. Then choose “General”.

### 2. General

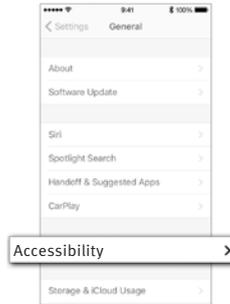


On the “General” screen, choose “Accessibility”.

### 3. Accessibility

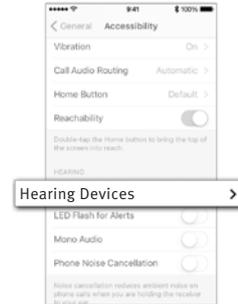


On the “Accessibility” screen, choose “Hearing Devices”.

### 4. Prepare

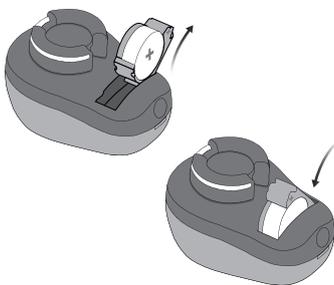


Open and close the battery drawer on the instrument(s), and place them close to your iPhone. The instrument(s) remains in pairing mode for 3 minutes.

### 5. Select

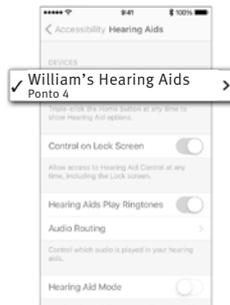

Your iPhone will detect the instrument(s) for pairing. Detected devices will appear in your iPhone list. Choose your instrument(s).

### 6. Confirm pairing

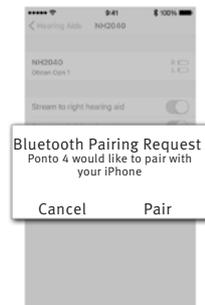


Confirm pairing. If you have two instruments, pairing confirmation is needed for each instrument.

## Pairing your remote control with your instrument(s)

---

### 1. Prepare remote control

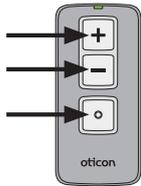

Press and hold all buttons for 5 seconds. The status LED will flash green twice.

### 2. Confirms ready for pairing




The status LED will then glow green for about 10 seconds to confirm the remote is ready for pairing.

### 3. Turn the instrument(s) OFF.




(open battery drawer)

### 4. Turn the instrument(s) ON.

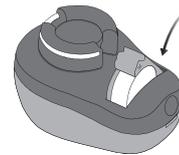

(close battery drawer)

### 5. Place Remote Control and the instrument(s) close to each other (max distance 20 cm)

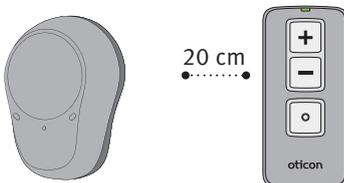


The instrument(s) will be ready for a first time connection with Remote Control for about 3 minutes.

### 6. Pair the remote control with the instrument(s)



Briefly press any key on the remote control to make it automatically pair with the instrument(s).

### 7. Connection established with instrument(s)

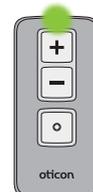


When pairing is complete the status light glows green for about 10 seconds.


## Pairing your TV Adapter with your instrument(s)

---

1. Make sure the TV Adapter is powered on.

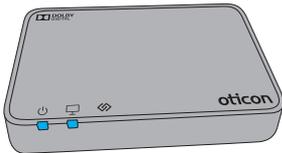


2. Turn instrument(s) OFF.

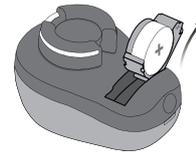

(open battery drawer)


3. Turn instrument(s) ON.

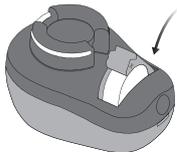

(close battery drawer)

4. The instrument(s) will now be ready for a first time connection with the TV Adapter for about 3 minutes

5. Place the instrument(s) close to or on top of the TV Adapter.

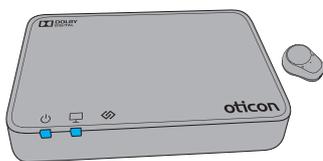

6. Connection established with instrument(s)

TV Adapter	Description
	Pairing completed

Pairing is completed when the power indicator and the TV indicator turn blue for about 30 sec.