

People First

People First is our promise to empower people to communicate freely, interact naturally and participate actively

**ClearSounds QH2
Bluetooth Hub & Phone Amplifier**

155841UK/03.15



**ClearSounds QH2
Bluetooth Hub & Phone Amplifier**

Introduction

The use of office phones with hearing instrument solutions can be difficult, because different brands of office phones use different proprietary/digital encoded signals.

However, with ClearSounds QH2 you can connect to any corded business (PBX, VoIP etc.) telephone through Bluetooth using your Streamer Pro and hearing aids as a personal wireless headset.

For up-to-date information, videos and user guides about the QH2 Bluetooth Phone Hub system please visit www.clearsounds.com



Using the QH2 with Streamer Pro

The Streamer (Streamer 1.4, Streamer Pro 1.0 or later) can be connected through Bluetooth to the QH2 Hub and Phone Amplifier, so that you can benefit from the wireless headset functionality.

All handling of a call can be done through the QH2 Hub and Phone Amplifier (or the office phone). Please refer to section 3 in the QH2 Quick Start Guide.

Once the included handset lifter is installed and the Streamer has been paired with the QH2 you will also be able to answer and to end a phone call directly through the Streamer.

How to get started

1. Read ClearSounds QH2 Quick Start Guide carefully
2. Connect QH2 to the office phone as described in section 1 in the Quick Start Guide
3. Pair the Streamer with QH2 as described in section 2 in the Quick Start Guide

Refer to the Streamer's Instructions for Use on how to use the Streamer as a personal headset.

For up-to-date information, videos and user guides about the QH2 Bluetooth Phone Hub system please visit www.clearsounds.com

For up-to-date information about the Streamer and ConnectLine please visit www.oticon.com/office



The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Oticon A/S is under license. Other trademarks and trade names are those of their respective owners.