

# Oticon On IFTTT guide

Get a notification 15 min before a Google Calendar event

## What you will need:

Opn hearing aid · iPhone 5 (or later) · Oticon ON app  
ON account · IFTTT account · Google account

## Initial Preparation

*How to set up your iPhone:*

- Go to app store on your iPhone
- Find the Oticon ON app in the apps section of the search results and click on the icon
- Find and press the Install button to download and install the app on your iPhone
- Open the Oticon ON app. Follow the instructions in the ON account creation section in the app

*How to create an IFTTT account:*

- Go to [www.ifttt.com](http://www.ifttt.com)
- Press 'Sign up' in the upper right corner
- Fill in your information and press 'Create account'

### Create your free account

You're only seconds away from doing more with the products you love.


**Your Email**

**Choose a Password**

**Create account**


1

Go to [www.ifttt.com](http://www.ifttt.com), log in with your IFTTT account and click on 'My Recipes' in the top bar, followed by 'Create a Recipe'.

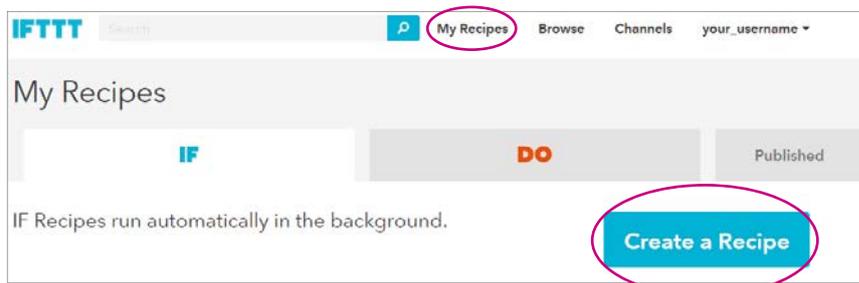


2

Click on the big blue 'this'

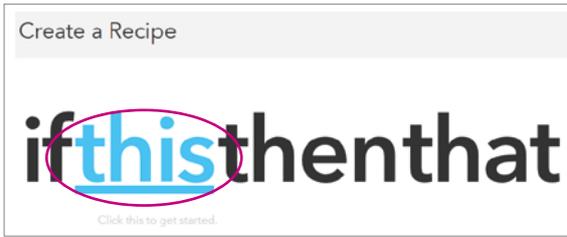


3

In the search field, type 'google calendar', and click on the 'Google Calendar' button

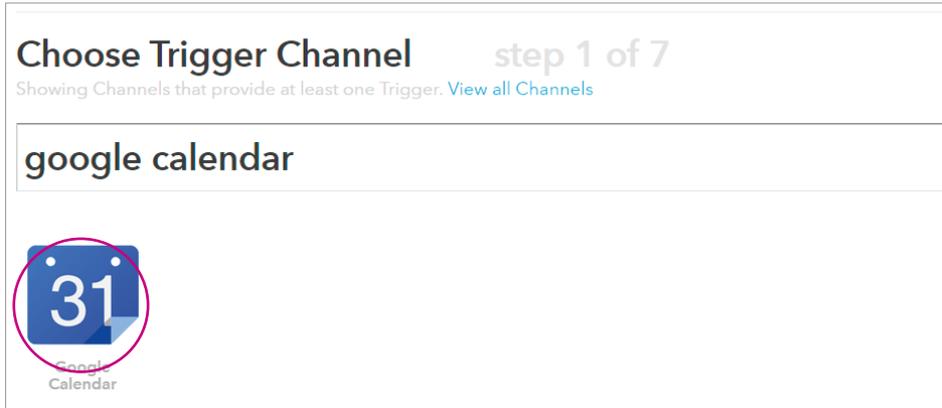

4

Click on eConnect. In the pop-up window: Log in with your Google account. Click on 'Continue to the next step'

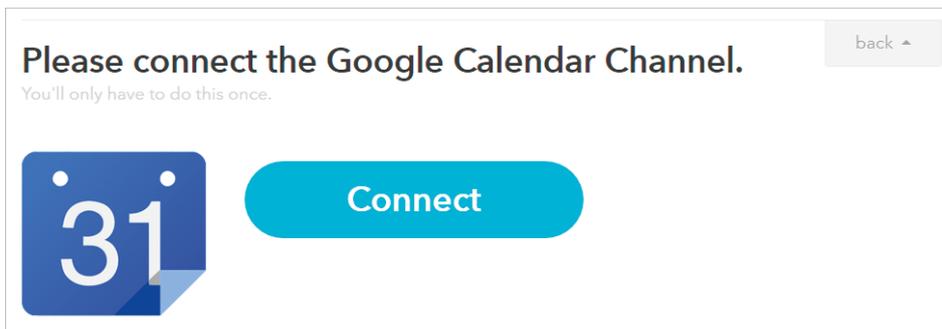


**Note:** Click 'allow' to provide IFTTT offline access to your Google calendar and choose the calendar which you want to get monitored.

 A screenshot of a Google account permission dialog box. It shows the IFTTT logo and asks for offline access. There are 'deny' and 'allow' buttons at the bottom.

5

Choose the trigger 'Event from search starts' by clicking on it.

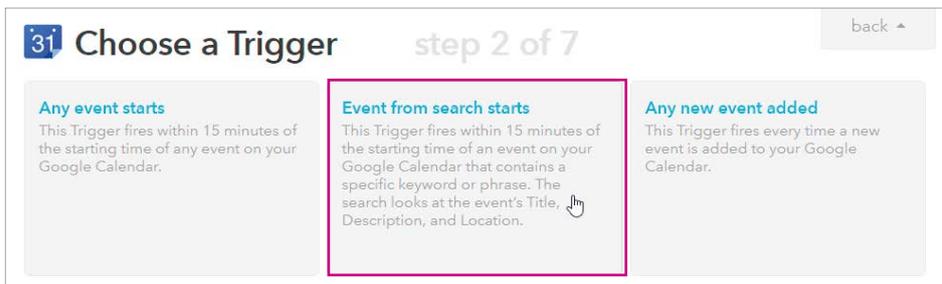

6

In the field 'Keyword or phrase', type the keyword you would like to use to mark the events to be reminded of in your hearing aid. For example, type: 'important'. Then click on 'Create trigger'.

**31 Complete Trigger Fields** step 3 of 7 back ^

Event from search starts

**31 Keyword or phrase**

**Create Trigger**

7

Click on the big blue 'that'



8

In the search field, type 'oticon', and click on the 'Oticon ON' logo.

**Choose Action Channel** step 4 of 7 back ^

Showing Channels that provide at least one Action. [View all Channels](#)



9

Click on Connect. In the pop-up window: Log in with your Oticon ON account.


**Please connect the Oticon Channel.** back ^

You'll only have to do this once.

 **Connect**


10

Click on the 'Play the text to speech' button.

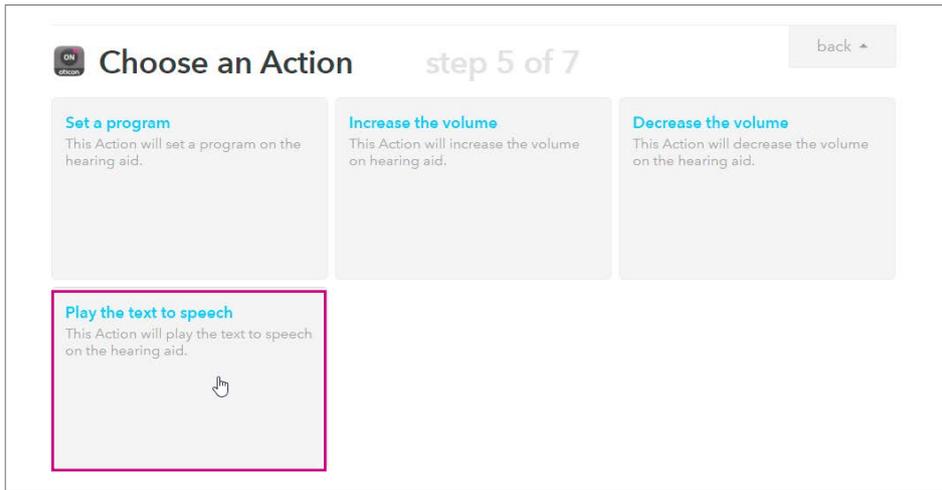

11

In the message window you can now enter the message you want to be sent. You can add the title or other information about the event in the message by clicking in the text box and then on the blue bottle inside the text box. For example: 'Your event {{Title}} starts {{Starts}}' Then click on 'Create Action'.

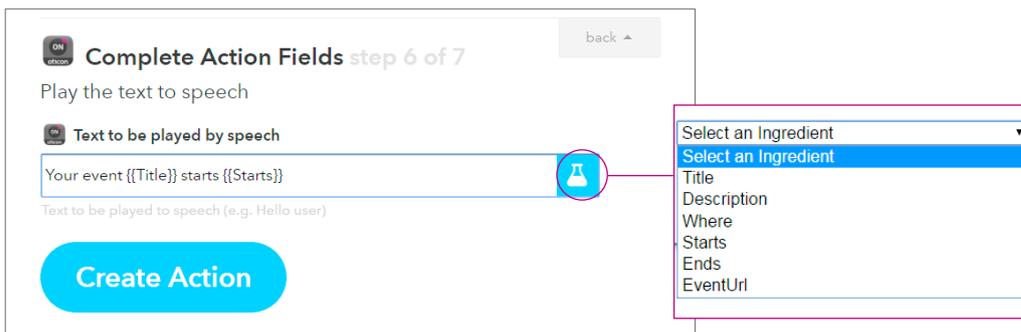

12

Finally click on 'Create Recipe' to create and activate the recipe which notifies you 15 min before a Google Calendar event.

