RELATION 2

Amplified telephone with
teleloop and light-signalling

User guide

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Bruksanvisning

 **Phonic Ear**[®]
Connected to life

Table of contents

Relation 2 user guide (English)	4
Relation 2 brugervejledning (Dansk)	30
Relation 2 bruksanvisning (Svenska)	56

To enable you to get your telephone working quickly and to use it safely, it is essential that you read the following safety instructions and the section on "Getting started".

▪ **Attention:**

Your telephone can be set to a high volume. To protect your hearing or the hearing of people with normal hearing, particularly of children, from any damage caused by excessively loud volume, please observe the following information:

1. Set the volume to a low level before you place the telephone handset on your ear.
 2. While using the device, set the volume to a level that is comfortable for you.
- Place the telephone on a non-slip surface.
 - Place the connecting cable safely in order to avoid the risk of accidents.
 - Never expose the telephone to any of the following influencing factors: sources of heat, direct sunlight or moisture.
 - Protect your telephone from damp, dust, aggressive liquids and steam.
 - Only insert the connecting cable to the designated sockets/jacks.
 - Only connect approved accessories to it.
 - Never
 - open the telephone casing yourself!
 - carry the telephone by the connecting cables!

Pacemaker safety

The Relation 2 is equipped with a permanent magnet for the "Autodetect" function of hearing aids.

Attention:

1. Before using the Relation 2 with a pacemaker or other medical device, consult your physician or the manufacturer of your pacemaker or other medical device.
2. If you have a pacemaker or other medical device, make sure that you are using this product in accordance with safety guidelines established by your physician or the pacemaker manufacturer.

Table of contents

Getting started

Pack content	8
Telephone connection	8
Installing the batteries	8

Controls

Controls on the front	9
Controls on the left / back / right	10
Buttons and their functions	11

Using the telephone

Making calls	12
Answering calls	12
Using the telephone speaker during a call	12
Setting the tone	12
Last number redial	12
Redial for the last 10 telephone numbers dialed	13
Store one of the last 10 telephone numbers dialed	13
Deleting the last telephone number dialed	13
Deleting all telephone numbers dialed	13
Handset volume	14
Switching the microphone on/off	14
Hold	14
Inserting a dialing pause (PABX)	14
Inquiry button (Recall time)	14
Support for hearing aid users	
Hearing aid compatibility	15
Autodetect function	15

Table of contents

Telephone book

Using the telephone book	16
Storing name/number	16
Placing a call with the telephone book	17
Editing an entry in the telephone book	17
Deleting an entry in the telephone book	17

One-touch dial keys

Allocating telephone numbers to one-touch dial keys	18
Using the one-touch dial keys	18
Deleting one of the one-touch dial keys	18

Calling number display

How it works	19
Reading and calling entries	19
Storing an entry	19
Deleting an entry	20
Deleting all entries	20

Settings

Setting the language	20
Setting the date format	20
Setting the time format	21
Setting the date & time	21
Setting the ring volume	22
Setting the ring melody	22
Setting the key tone	22
Setting the contrast	22
Setting the recall time	23
Setting the dynamic font	23

Table of contents

Setting the dialing mode	23
Setting the call waiting	23
Setting the Auto On Hook	24
Setting the AUTO AMP function	24
Setting the MIC AMP function	24
Additional ports	
Connection jack for vibrating pillow and/or modules	24
Headset port	24
Headphone port	24
Resetting the telephone to the factory settings	25
Power failure operation	25
If your telephone doesn't work	
General information	26
No number appears when the phone rings	26
The telephone doesn't work	26
Warranty	27
Appendix	
Environmental note / Waste disposal	28
Maintenance and care	29
Technical data	29
Test and approvals	29

Getting started

Pack content

Please check if all following components are included:

- Telephone base unit
- Telephone handset
- Handset cord
- Telephone cable
- 7.5V plug-in PSU
- 4 pcs. Alkaline 1.5V AA
- User guide

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Telephone connection

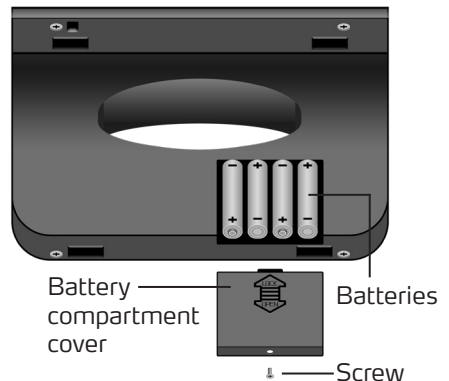
1. Insert the handset cord (spiral-shaped cable) into the jack marked with a receiver symbol on the left or right side of the base unit and into the handset.
2. Insert the smaller plug for the telephone cable into the "LINE" jack (back of the base unit). Insert the larger plug into the socket for your telephone line.
3. Insert the plug-in mains adapter cable into the "DC 7.5V" jack. Plug the mains adapter into a 230 V mains socket.

Warning!

Don't place the telephone close to televisions, fans, etc. so as to minimize the risk of interference. Don't place the telephone in direct sunlight or close to a source of heat. Only use the original telephone cable or a comparable cable with an international configuration.


Installing the batteries

1. Remove the battery compartment cover by unscrewing the screw and sliding the cover off.
2. Insert 4 fresh Alkaline 1.5V AA batteries while observing the +/- polarity marks in the battery compartment.
3. Replace the battery compartment cover and secure it with the screw.
4. Connect the telephone cable to the wall outlet for telephone.


Controls

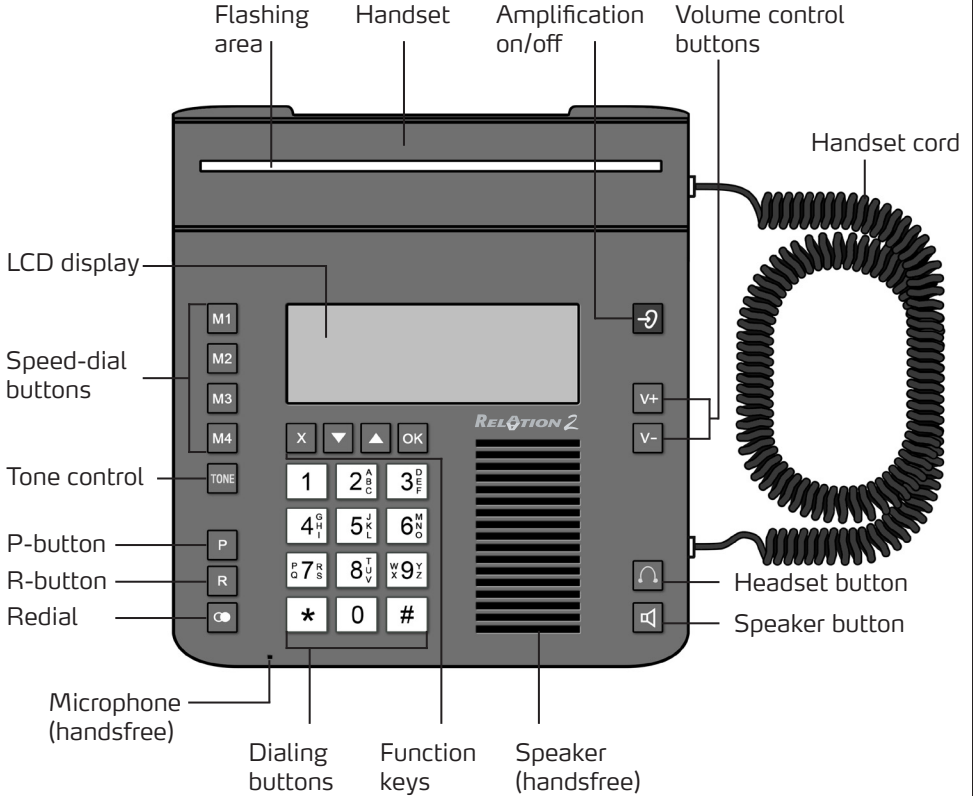Controls on the front


Controls

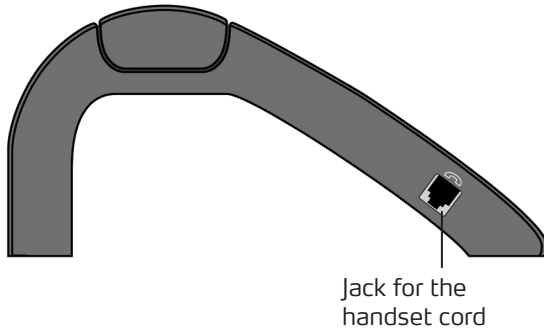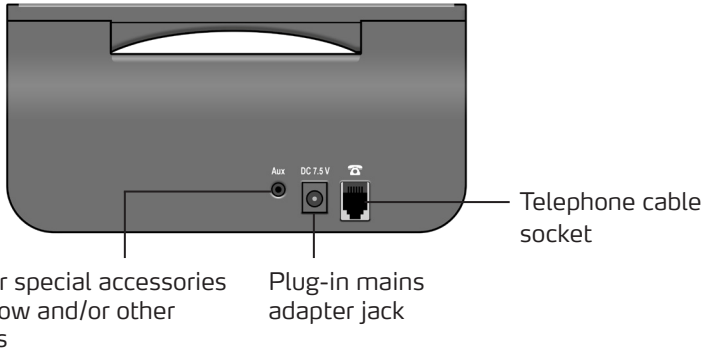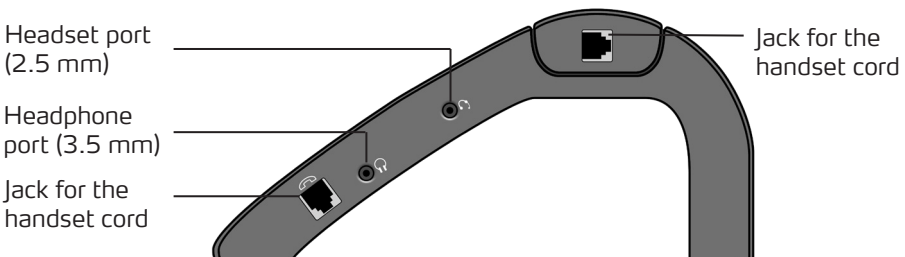
Controls on the left

Controls on the back

Controls on the right

Buttons and their functions

-  Speed-dial buttons
-  Tone control during a phone call
-  Pause key: Insert pause while dialing a number
-  Inquiry button, e.g. for putting calls through if the telephone is connected to a telephone system
-  Redial
-  Back/delete last entry/phonebook/switching the microphone on/off
-  Arrow keys down/up: Scroll through menu
-  OK button: Confirm selection, select menu and hold button
-  To enter the digits 0-9
-  To enter *
-  To enter #
-  Amplification on/off
-  Volume control buttons
- 
-  Headset speaker:
- Making a phone call over a connected headset
- Engage the line, hang up
-  Speaker button:
- "Handsfree talking": engage the line, hang up
- Enable/disable "conference" mode during a call

Using the telephone

Making calls

1. Lift up the handset or press the  button. If you would like to make a phone call over a connected headset press .
2. Dial the telephone number.
3. In order to end the call, replace the handset or press either the  button or the .

Answering calls

1. The telephone rings.
2. Lift up the handset or press the  button. If you would like to answer a phone call over a connected headset press .
3. In order to end the call, replace the handset or press either the  button or the .

Using the telephone speaker during a call

1. Press the  button to activate the telephone speaker during a call.
2. You can now replace the handset and continue the call over the telephone speaker and integrated microphone.
3. To deactivate the telephone speaker again lift up the handset.

Note: A call is ended if you press the  button while the handset is on the cradle.

Setting the tone

1. Press the  button to adjust the tone during a phone call.
2. The tone can be adjusted in four steps by pressing the .

Note: The setting of the tone works only if the handset is lifted up or with a connected headset. During a call over the telephone speaker the tone can not be adjusted.

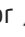
Last number redial

1. Press the  button. The last dialed number will be shown on the display.
2. Lift up the handset or press either the  button or the  button if a headset is connected. The number is redialed.

Using the telephone

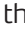
Redial for the last 10 telephone numbers dialed

The last 10 telephone numbers dialed will be stored in the telephone automatically.

1. Press .
2. Use  or  to page between the entries.
3. Once the number you are looking for appears in the display, lift the handset or press either the  button or the  button if a headset is connected. The number you have selected will be dialed automatically.

Note: The handset must be on the cradle while you are selecting the telephone number you want to redial.

Store one of the last 10 telephone numbers dialed

1. Press .
2. Use  or  to page between the entries.
3. Once the number you are looking for appears in the display, press . Use  or  to page to "Store".
4. Choose "Store" to store the chosen entry and press .
5. "Number:" and the phone number will be shown in the display. Press  to confirm.
6. Now "Name:" will be shown and you can enter a name for the telephone number.
7. After you have entered the name, press .
8. Now you can select a ringtone for your entry.
9. Press  to confirm and save your entry.

Deleting the last telephone number dialed

1. Press .
2. Use  or  to page between the entries.
3. Once the number you are looking for appears in the display, press . "Erase" will appear in the telephone display.
4. Press  to delete the chosen entry.

Deleting all the telephone numbers dialed

1. Press .
2. Press  and use  or  to page between "Erase", "Erase all" and "Store".
3. Choose "Erase all" if you want to delete all entries and press .
4. "Erase all?" will appear in the display and press  to confirm.

Using the telephone

Handset volume

V+

V-

The handset volume can be adjusted during a call with the **V+** and **V-** push buttons on the front side of the telephone.

Additional amplification can be switched on by pressing the **[A]** button. When additional amplification is activated, the handset volume can be raised to up to 30 dB. If the amplification is switched on, the **[A]** button will light up. The additional amplification will be switched off by pressing the **[A]** button again.

Switching the microphone on/off

You can switch the microphone on your telephone on and off during a telephone call. Procedure for switching off the microphone:

1. Press the button **[X]**. The microphone is now deactivated.
2. Press the button **[X]** again to switch the microphone back on.

Hold

This function enables you to interrupt a conversation temporarily without ending the call. Press the **[OK]** button to interrupt the conversation temporarily.

Inserting a dialing pause (PABX)

Press the **[P]** button to interrupt dialing for 1 seconds.

Inquiry button (Recall time)

This function is required in order to activate certain T-Net features. If you press the **[R]** button, the telephone line is interrupted for a certain length of time.

Hearing aid compatibility

Relation 2 is an HAC telephone, which means it is Hearing Aid Compatible. It is designed with the hearing-impaired user in mind although it works well for anyone who wants exceptionally clear sound reception.

Relation 2 has a built-in teleloop in the handset and works with hearing aids with a telecoil programme (programme T). When you set the hearing aid to programme T (telecoil), the hearing aid's microphone is switched off and background noise is automatically excluded.

When your hearing aid is set to programme M (microphone), it is important to hold the handset close to the hearing aid's microphone (please note that background noise will not be excluded when the hearing aid is set to programme M). To avoid feedback, you might need to turn the handset a little.

Some hearing instruments are also equipped with an MT programme, which means that the microphone and the telecoil can be used at the same time.

Various types of assistive listening devices such as FM transmitters can be connected using the headphone port.

Autodetect function

The telephone handset is equipped with a permanent magnet for the autodetect function of hearing aids. If the hearing aid is equipped with the autodetect function it will automatically switch to telephone mode if the hearing aid comes in close proximity to the handset speaker.

Telephone book

Using the telephone book

Names and numbers can be stored in the telephone book. Once a telephone number is stored, you can call that number by pressing a minimum number of keys. Up to 30 telephone numbers can be stored in the telephone book. Each telephone number can contain up to 24 digits.

Storing name/number

To make new entries in the telephone book, proceed as follows:

1. Enter the desired telephone number on the keypad. Press **X** to delete the last digit entered.
2. Press **OK**. "Number:" and the entered telephone number will be shown in the display.
3. Press **OK** again and enter the desired name of the telephone number on the keypad.
4. Press the corresponding key as frequently as necessary to select the desired letter (see table below). Press **X** to delete the last letter entered.
5. Press **OK** to confirm your name entry and switch to the selection of the ringtone for the entry.
6. Use **▼** or **▲** to page between the ringtones.
7. Press **OK** to complete the programming process.

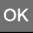
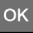
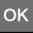
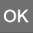
Key	Letter / character
0	0
1	1 . , ; ' " &
2	a b c 2 å æ ä ß ç
3	d e f 3 é
4	g h i # 4
5	j k l / 5
6	m n o ' 6 ñ ó ö ø
7	p q r s 7
8	t u v ? 8 ü
9	w x y z 9
#	[Space]

Note: If you would like to shift the case between upper and lower, while entering letters, press the * key.

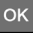
Placing a call with the telephone book

1. Press .
2. Use  or  to select the desired entry.
3. Lift the handset or press either the  button or the  button to dial the number you have selected.

Editing an entry in the telephone book

1. Press . The first entry in the telephone book will be shown.
2. Use  or  to select the desired entry you would like to edit.
3. Press  after you have selected the desired entry.
"Edit" will be shown in the telephone display.
4. Press  and the number of the entry will be shown on the telephone display.
5. Press  to delete a number or press the desired number on the keypad.
6. Press  to confirm your number entry and switch to the entry of the name.
7. Press  to delete a letter or press the corresponding key as frequently as necessary to select the desired letter (see table on page 16).
8. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
9. Use  or  to page between the ringtones.
10. Press  to complete the programming process.

Deleting an entry in the telephone book

1. Press . The first entry in the telephone book will be shown.
2. Use  or  to select the desired entry you would like to delete.
3. Press  after you have selected the desired entry.
"Edit" will be shown in the telephone display.
4. Use  or  to select "Erase".
5. Press  and the selected entry will be deleted.

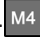
One-touch dial keys

Allocating telephone numbers to one-touch dial keys

The 4 speed-dial keys can be programmed with any telephone number.

1. Enter the desired telephone number on the keypad. Press  to delete the last digit entered if necessary.
2. Press the desired one-touch dial key for longer than 3 seconds. "Number:" and the entered telephone number will be shown in the display.
3. Press  and enter the desired name of the telephone number on the keypad.
4. Press the corresponding key as frequently as necessary to select the desired letter (see table on page 16). Press  to delete the last letter entered if necessary.
5. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
6. Use  or  to page between the ringtones.
7. Press  to assign the selected number to the corresponding location.

Using the one-touch dial keys

Take the handset from the cradle or press either the  button or the  button. If you press one of the keys marked ... the telephone number stored for that key will be dialed automatically.

Deleting one of the one-touch dial keys

1. Press . The first entry in the telephone book will be shown.
2. Use  or  to page to the desired one-touch dial entry in the phonebook.
3. Press .
4. "Edit" will be shown in the telephone display.
5. Use  or  to page between "Edit" and "Erase".
6. Press  after you have chosen "Erase".
7. The telephone number is deleted from the one-touch dial key and from the telephone book.

Calling number display

How it works

The number of the person calling you will appear in the telephone display. The telephone supports both DTMF and FSK. Your telephone will store the last 20 calling numbers automatically. If the memory is full, the oldest entry will be deleted.

Important! Calling number display will only work if this service is supported by your telecom supplier. Please inquire with your telephone company.

Reading and calling entries

The last 20 numbers recognized will be automatically stored in the phone.

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Use ▼ or ▲ to page between the entries. The first 12 digits of the telephone number or a stored name will appear in the display. Additionally the date and the time of the call record will be shown.
3. Once the desired telephone number appears in the display, lift the handset or press either the  button or the  button. The number you have selected will be dialed.

Note: The handset must be resting on the cradle when you select the telephone number.

Storing an entry

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Use ▼ or ▲ to page between the entries.
3. Press  if the desired entry will be shown. "Store" will be shown in the telephone display.
4. Press . "Number:" and the telephone number will be shown in the display.
5. Press  again and enter the desired name of the telephone number on the keypad.
6. Press the corresponding key as frequently as necessary to select the desired letter (see table on page 16). Press  to delete the last letter entered.
7. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
8. Use ▼ or ▲ to page between the ringtones.
9. Press  to complete the programming process.

Calling number display and settings

Deleting an entry

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Use ▼ or ▲ to page between the entries.
3. Press **OK** if the desired entry will be shown.
"Store" will be shown in the display.
4. Use ▼ or ▲ to page between "Store", "Erase" and "Erase all".
5. Choose "Erase" and press **OK** to delete the chosen entry.

Deleting all entries

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Press **OK**. "Store" will be shown in the display.
3. Use ▼ or ▲ to page between "Store", "Erase" and "Erase all".
4. Choose "Erase all" and press **OK**. "Erase all?" will be shown in the display.
5. Press **OK** again to confirm and delete all entries.

Setting the language

1. Press **OK**. "Set Display" will be shown in the display.
2. Press **OK** again. "Set Language" will be shown in the display.
3. Press **OK** to enter the language selection.
4. Use ▼ or ▲ to page to the desired language.
5. Press **OK** to confirm your selection.

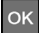
Setting the date format

1. Press **OK**. "Set Display" will be shown in the display.
2. Press **OK** again to enter the display setting menu.
3. Use ▼ or ▲ to page to "Date Format".
4. Press **OK** to enter the date format selection.
5. Use ▼ or ▲ to page between "MM/DD" (MonthMonth-DayDay) and "DD/MM" (DayDay-MonthMonth).
6. Press **OK** to confirm your selection.

Setting the time format


1. Press . "Set Display" will be shown in the display.
2. Press  again to enter the display setting menu.
3. Use  or  to page to "Hour Format".
4. Press  to enter the time format selection.
5. Use  or  to page between "12 hour clock" and "24 hour clock".
6. Press  to confirm your selection.

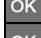Setting the date & time

1. Press . "Set Display" will be shown in the display.
2. Use  or  to page to "Date & Time".
3. Press  to enter the date and time menu. "Time:" will be shown in the display and the first number of the hours blinks.
4. Use the keypad to enter the hours.
5. The telephone switches to the minutes after the enter of the hours and the first digit blinks.
6. Use the keypad to enter the minutes.
7. Press . The telephone switches to the date entry and the first digit blinks.
8. Use the keypad to enter the numbers for the date. It depends on the setting of the date format if you start with the day followed by the month or vice versa. The telephone switches to the next digit after each keypad entry.
9. Press . "YEAR:" and the set number of the year will be shown on the display. Additionally the third digit blinks.
10. Use the keypad to enter the desired number of the year.
11. Press  to confirm your entries.

Settings

Setting the ring volume

1. Press . "Set Display" will be shown in the display.
2. Use  or  to page to "Set Phone".
3. Press  to enter the phone setting menu. "Set Ring" will be shown in the display.
4. Press . "Ringer volu." will be shown in the display.
5. Press  to enter the ring volume selection.
6. Use  or  to page between "Ringer OFF" and 5 volume levels.
7. Press  to confirm your selection.

Setting the ring melody

1. Press . "Set Display" will be shown in the display.
2. Use  or  to page to "Set Phone".
3. Press  to enter the phone setting menu. "Set Ring" will be shown in the display.
4. Press . "Ringer volu." will be shown in the display.
5. Use  or  to page to "Ringer tone".
6. Press  to enter the ring melody selection. The set ring tone will be shown.
7. Use  or  to page between 10 different melodies. You hear the ring tone.
8. Press  to confirm your selection.

Setting the key tone

1. Press . "Set Display" will be shown in the display.
2. Use  or  to page to "Set Phone".
3. Press  to enter the phone setting menu. "Set Ring" will be shown in the display.
4. Press . "Ringer volu." will be shown in the display.
5. Use  or  to page to "Keypad Tones".
6. Press  to enter the key tone menu.
7. Use  or  to page between "ON" and "OFF".
8. Press  to confirm your selection.

Setting the contrast

1. Press . "Set Display" will be shown in the display.
2. Press  again. "Set Language" will be shown in the display.
3. Use  or  to page to "Set Contrast".
4. Press  to enter the contrast selection. The set contrast level will be shown.
5. Use  or  to page between 8 different contrast levels.
6. Press  to confirm your selection.

Setting the recall time

1. Press **OK**. "Set Display" will be shown in the display.
2. Use **▼** or **▲** to page to "Set Phone".
3. Press **OK** to enter the phone setting menu. "Set Ring" will be shown in the display.
4. Use **▼** or **▲** to page to "Set Flash".
5. Press **OK** to enter the recall time selection. "300ms" will be shown in the display.
6. Use **▼** or **▲** to page between "100ms", "300ms" and "600ms".
7. Press **OK** to confirm your selection.

Setting the dynamic font

1. Press **OK**. "Set Display" will be shown in the display.
2. Press **OK** again to enter the display setting menu.
3. Use **▼** or **▲** to page to "Dim:X--->x".
4. Press **OK** to enter the dynamic font menu.
5. Use **▼** or **▲** to page between "ON" and "OFF".
6. Press **OK** after you have chosen the desired setting.

Setting the dialing mode

1. Press **OK**. "Set Display" will be shown in the display.
2. Use **▼** or **▲** to page to "Set Phone".
3. Press **OK** to enter the phone setting menu. "Set Ring" will be shown in the display.
4. Use **▼** or **▲** to page to "Dialing Mode".
5. Press **OK** to enter the dialing mode setting menu.
6. Use **▼** or **▲** to page between "Tone" and "Pulse".
7. Press **OK** after you have chosen the desired setting.

Setting the call waiting

1. Press **OK**. "Set Display" will be shown in the display.
2. Use **▼** or **▲** to page to "Set Phone".
3. Press **OK** to enter the phone setting menu. "Set Ring" will be shown in the display.
4. Use **▼** or **▲** to page to "Call waiting".
5. Press **OK** to enter the call waiting setting menu.
6. Use **▼** or **▲** to page between "ON" and "OFF".
7. Press **OK** after you have chosen the desired setting.

Settings and additional ports

Setting the Auto On Hook

1. Press **OK**. "Set Display" will be shown in the display.
2. Use **▼** or **▲** to page to "Set Phone".
3. Press **OK** to enter the phone setting menu. "Set Ring" will be shown.
4. Use **▼** or **▲** to page to "AUTO ON HOOK".
5. Press **OK** to enter the auto on hook setting menu.
6. Use **▼** or **▲** to page between "ON" and "OFF".
7. Press **OK** after you have chosen the desired setting.

Setting the Auto AMP function

1. Press **OK**. "Set Display" will be shown in the display.
2. Use **▼** or **▲** to page to "Auto AMP".
3. Press **OK** to enter the automatic amplification menu. "OFF" will be shown in the display.
4. Use **▼** or **▲** to page between "ON" and "OFF".
5. Press **OK** after you have chosen the desired setting.

Setting the Mic AMP function

1. Press **OK**. "Set Display" will be shown in the display.
2. Use **▼** or **▲** to page to "Mic AMP".
3. Press **OK** to enter the microphone amplification menu. "OdB" will be shown in the display.
4. Use **▼** or **▲** to page between "+4dB", "OdB" and "-4dB".
5. Press **OK** after you have chosen the desired setting.

Connection jack for vibrating pillow and/or modules

Your telephone is equipped with a connection jack for a vibrating pillow or other additional signaling modules from Phonic Ear. You can take advantage of this option to signal an incoming call not only with a very loud ring and the very bright flashing light, but also with an optionally available vibrating pillow and/or other assistive listening devices from Phonic Ear.

Headset port

Standard headset port with a 2.5 mm jack.

Headphone port

Output for connecting headphones, an induction loop or such like.

Resetting and power failure operation

Resetting the telephone to the factory settings

Setting the telephone to the default settings

1. Press **OK**. "Set Display" will be shown in the display.
2. Enter 042200011 over the keypad.
3. Press **OK**. You hear a signal tone.
4. Disconnect the small PSU plug from the DC connection jack at the backside of the telephone.
5. Please wait three seconds and reconnect the small PSU plug with the DC connection jack.
6. The telephone is set to the default settings now.

Default settings:

One-touch dial keys:	Empty
Redial memory:	Empty
Calling number display memory (CLIP):	Empty
Telephone book:	Empty
Recall time:	300 ms
Dialing mode:	Tone
Ring tone:	2
Keypad tones:	Switched on
Volume:	Level 4

Power failure operation

Should there be a power failure, you can still use your telephone. However, the flashing light will not work.

If you would like to use the telephone during a power failure, you can put 4 batteries (Alkaline 1.5V AA) into the telephone.

Note: During a power failure, the telephone's flashing light will not work even if you have put in batteries.

The dead battery symbol  will appear in the display if the batteries are dead.

If your telephone doesn't work...

General information

Make sure that the telephone cable isn't damaged and is correctly inserted. Disconnect any additional equipment, extension cables and other telephones. If the telephone works then, this indicates that the fault is caused by another piece of equipment. Try to connect your telephone to a neighbor's line, for example. If it works there, this indicates that there may be a fault on your telephone line. Call the service provider to report the fault.

No number appears when the phone rings

- Calling number display is a function that has to be supported by your service provider.
- If no calling number appears in the display, this means that no calling number information is available or that the caller has an unlisted number. This can also happen when a call is received from a telephone exchange.
- If the telephone is connected to an extension, the calling number display function may not work.

The telephone doesn't work

- Make sure that the mains adapter is correctly inserted into the telephone and into the mains socket.
- Make sure that the telephone cable is correctly inserted into the telephone and into the telephone socket.
- Connect another telephone to your telephone line. If it works, it indicates that your telephone is probably faulty.
- Only use the original cable or a telephone cable with an international configuration.

Warranty

A two-year limited warranty is issued to the original consumer of this product by Phonic Ear A/S.

Time period of warranty

This warranty will come into effect upon the date of the original purchase of this equipment, and will remain in effect for two calendar years from that date as long as the instrument remains the property of the original consumer.

What is covered by this warranty

Any electronic component which, because of workmanship, manufacturing, or design defects, fails to function properly under normal use during the lifetime of this warranty will be replaced or repaired at no charge for parts or labour when returned to the factory service centre. Transportation costs are to be met by the customer. If it is determined that repair is not feasible, the entire unit may be replaced with an equivalent unit upon mutual agreement of the manufacturer and customer.

What is not covered by this warranty:

1. Malfunctions resulting from abuse, neglect or accident.
2. Peripheral accessories as itemised within the product brochure, when such items are returned after 90 days from the original purchase.
3. Batteries.
4. Instruments connected, installed, used or adjusted in a manner contrary to instructions provided by the manufacturer.
5. Consequential damages and damages resulting from delay or loss of this instrument. The exclusive remedy under this warranty is strictly limited to repair or replacement as herein provided.
6. Products damaged in transit unless investigated by the shipper and returned to the warrantor with the investigation report.

Phonic Ear A/S reserves the right to make changes to the design or construction of any of its instruments at any time without incurring any obligation to make any changes whatsoever on units previously purchased. This warranty is in lieu of all other expressed warranties. All expressed and implied warranties will terminate upon the expiration of this written warranty. No representative or person is authorised to represent or assume for us any liability in connection with the sale or use of our products other than as set forth above.

The above warranty does not affect any legal rights that you might have under applicable national legislation governing the sale of consumer goods. Your distributor may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult your distributor for further information.

What to do if you need service

If you require service under the terms of this warranty, carefully package the instrument to prevent damage and return it to your distributor (see information on back cover). Please provide your receipt, a detailed description of the problem, your full name, billing address and telephone number.

Appendix

Environmental note / Waste disposal

Electrical and electronic equipment (EEE) contains materials, components and substances, that could be dangerous or detrimental to human health and the environment if waste electrical and electronic equipment (WEEE) is not disposed of correctly.

Products marked with the crossed-out dustbin shown below are electrical and electronic equipment. The crossed-out rubbish bin indicates that waste electrical and electronic equipment may not be disposed of with unsorted household waste and that it must be collected separately.

In developing and manufacturing your Phonic Ear product, we used high-quality materials and components that can be reused. Thus, when disposing of your waste electrical and electronic equipment, you must use a collection system designated for this purpose, regardless of whether you dispose of your waste at your local waste site or it is collected from your home. Please contact your local authority for further information.

Batteries contain substances that can be harmful to human health and the environment if not handled correctly.

Batteries are marked with the crossed-out rubbish bin symbol illustrated below. This symbolises that exhausted batteries must not be disposed of together with unsorted household rubbish, but are to be collected separately.

Some batteries are also marked with the chemical symbol Hg (mercury), Cd (cadmium) or Pb (lead). These substances are particularly harmful, so it is important that they are collected and disposed of via an established recycling scheme.

This ensures that your batteries are reused in compliance with regulations and do not damage the environment.

Therefore, when disposing of exhausted batteries, make sure that you utilise an established recycling scheme. This might involve delivering them to a designated collection point or recycling centre, or they may be collected directly from your address. For further details, please contact your local authorities.

Maintenance and care

The telephone is maintenance-free. If the telephone becomes dirty, wipe it occasionally with a soft, damp cloth. Never use alcohol, thinner or other organic solvents. The telephone should never be exposed to sunlight for a long time and should also be protected from intense heat, moisture or strong mechanical vibrations.

Technical data

Power supply: Plug-in PSU:
Primary 100-240 V 50 Hz / 0.2A Max -
Secondary 7.5 V DC / 300 mA

Batteries: 4 pcs. Alkaline 1.5V AA

Dimensions (WxDxH): 178 mm x 210 mm x 96 mm

Weight: 900 g

Technical specifications subject to change without notice.

Test and approvals

This telephone satisfies the following EU directives:

- 2002/95/EC RoHS directive
- 2002/96/EC WEEE directive
- 1999/5/EC R&TTE directive
- 2004/108/EC EMC directive
- 2006/95/EC Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the telephone. CE compliance declarations are available on the Internet at www.phonicear.com.

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© 2016 Phonic Ear A/S