

Neuro 2 Swim Kit

Quick Reference Guide



The Swim Kit is a waterproof solution for Neuro 2 sound processor allowing water immersion for 2 hours at depths up to 3 meters. This accessory keeps your processor dry during water activities, such as swimming, boating or kayaking.


Intended Use

The Swim Kit is an accessory that provides extended protection to the Neuro 2 sound processor when used in water or humid and dusty environments.


The Swim Sleeve is designed to be used for one year or 50 times depending on which one is reached first. Always carefully inspect the Swim Sleeve before and after use. If you observe any changes to the sleeve material or tightness, stop using the sleeve and order new ones (Swim Sleeves Kit).

The information in this Quick Reference Guide does not replace the Swim Kit Instructions for Use. Make sure to familiarise yourself with the Swim Kit by reading the Instructions for Use.


How to Insert the sound processor into the Swim Sleeve

 **Note:** Always use rechargeable batteries to power the sound processor when using the Swim Kit. Zinc-Air batteries rely on fresh air supply to produce power and therefore cannot be used inside a Swim Sleeve.


- 1 Open the lid of the Swim Sleeve.
- 2 Inspect the Swim Sleeve to ensure it is free from any dirt, dust and hair fragments in the opening for the Swim Antenna connector, and the opening and lid for the processor insertion.
- 3 Insert the sound processor fully into the Swim Sleeve. During the insertion, bend the sleeve and carefully lead the sound processor hook to its place to avoid it from sticking to the sides.

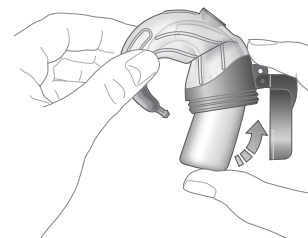


4 Ensure the sound processor is fully inserted so that its antenna connector is perfectly aligned with the opening in the Swim Sleeve.

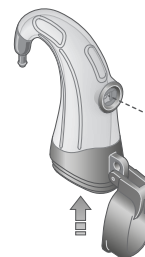

5 Hold the angled Swim Antenna plug with the cable perpendicular to the Swim Sleeve and insert the plug fully into the sound processor and sleeve. The red seal ring must touch the sleeve.

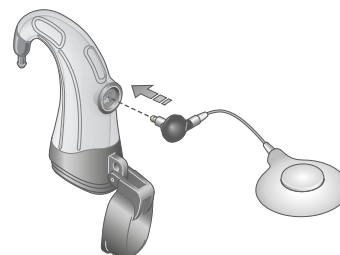

6 Rotate the plug 90 degrees clockwise to lock it in place. The antenna cable should end up being parallel to the sleeve.

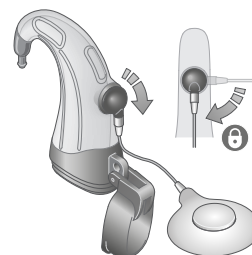

7 Close the lid of the Swim Sleeve.

8 Place the system on your ear.



The statements of this Quick Reference Guide are believed to be true at the date of the publication. The pictures are not contractually binding. Availability of the products depends on the local registration.

Oticon Medical (NEURELEC) reserves the right to make changes to the design, characteristics and models without prior notice. The only warranty Oticon Medical (NEURELEC) makes, is the express written warranty extended on the sale or rental of its products.

This Medical Device must be administrated/applied/implanted by a healthcare professional. Performance of the hearing devices is dependent on individual circumstances and may not be appropriate for all conditions. Always read and follow the instructions for use.

Oticon Medical
NEURELEC
2720 Chemin Saint-Bernard
06220 Vallauris – France
Tel: +33 (0)4 93 95 18 18
FAX: +33 (0)4 93 95 38 01
info@oticonmedical.com

CE 0459 (2020)

Oticon Medical (Australia)
Suite 4, Level 4, Building B,
11 Talavera Road, North Ryde
NSW 2113, Australia
info@oticonmedical.com.au

Audmet New Zealand Limited
(Trading as Oticon Medical)
Level 13, 142 Lambton Quay
Wellington 6011, New Zealand
info@oticonmedical.co.nz

223097AU - version A / 2020.08