PHILIPS

HearSuite

Fitting System

2020.1

Instructions for Use

PHILIPS

HearSuite

Fitting System

UK

2020.1

Instructions for Use

Table of contents

1. Introduction	4
2. Installation	5
3. System requirements	6
4. Getting started with Philips HearSuite	6
5. Philips Firmware Updater	9
6. HearSuite Updater	10
7. Warnings	10
8. Disclaimer	14

Microsoft, Windows 10, Windows 8, Windows 7 and Explorer are registered trademarks of Microsoft Corporation in the U.S. and/or other countries.

HI-PRO is a registered trademark of GN Otometrics A/S in the U.S. and in other countries. Noah is a registered trademark of HIMSA II K/S in the U.S.

Outside the U.S., Noah is a trademark of HIMSA II K/S.

NOAHlink is a registered trademark of HIMSA II K/S in Denmark.

Outside Denmark, NOAHlink is a trademark of HIMSA II K/S.

BLUETOOTH is a registered trademark of Bluetooth SIG, Inc.

EXPRESSLINK is a registered trademark of Sonic Innovations, Inc. in the U.S. and in other countries.

FittingLINK is a product of SBO Hearing A/S

In addition, system and product names used in this document are, in general, trademarks or registered trademarks of their respective developers or manufacturers.

However, the ™ or ® marks are not used in all cases in this document.

1. Introduction

These instructions for use are valid for Philips HearSuite 2020.1.

Philips HearSuite is a fitting software used for Philips HearLink hearing instrument families and models.

A printed version of this document can be obtained from your local distributor. If you have additional questions about the use of the fitting software, please contact your local distributor.

If you have additional questions about the use of the fitting software, please contact your local distributor.

Note

For better readability, HearSuite is written in the manual instead of Philips Philips HearSuite 2020.1.

Intended use

The fitting software is intended for the fitting and updating of hearing solutions.

The fitting software can facilitate access to real-ear measurement.

Indications for use

No indication for use.

Intended user

Hearing care professional.

Use environment

Clinical setting.

Contraindications

No contraindications.

Clinical benefits

See clinical benefits of the hearing aid.

2. Installation

This USB flash media contains several different software modules:

- Philips HearSuite
- Philips Firmware Updater
- HearSuite Updater
- SoundStudio

Installation of the software will install HearSuite. If you already have HearSuite installed, it will automatically be updated.

Preparing your PC

Before starting the installation, please make sure that your PC hardware and software meet the minimum specified system requirements and that your Windows system is updated to the latest published service pack. Please also make sure that you have administrator rights on the PC. Some virus software may compromise the installation process and you might have to deactivate your virus scanner to be able to run the installation.

Installation

Insert the USB flash media into a USB port. Please start your file explorer, click on the USB flash media and start **setup.exe**. After starting the installer, please follow the instructions on the screen.

UK 3. System requirements

Please make sure your PC complies with the following minimum system requirements.

HARDWARE	Processor	2 GHz or faster
	RAM	4 GB or more
	Free Hard Disk Space	8 GB minimum
	Graphics	1280 x 1024 pixels
	Ports	USB 2.0 for USB installation and programming devices
	Sound	(for SoundStudio) 5.1 surround sound card
SOFTWARE	Operating System	Windows® 10 (32/64 bit), all editions except ARM platform and Windows® 10 S Windows® 8 (32/64 bit), all editions except RT Windows® 7 SP1 (32/64 bit), all editions
	Noah	Noah 4 (minimum) Note: Noah based Office systems must be HIMSA certified

Note: Installation of software on computers that do not meet specific system requirements noted above may result in system failures.

4. Getting started with Philips HearSuite

To start a fitting session, enter or open an existing client in your office system and start HearSuite.

In the following chapter, the most important steps of fitting hearing instruments are explained. Please refer to our training material to get more insight into HearSuite fitting software.

Client Data

Client data, preferred language, fitting rationale, audiogram and REM data can be entered here. Verify and update

audiometric parameters to ensure correct gain prescription. This is especially relevant for the fitting of hearing instruments for infants and young children.

To detect and connect to the hearing instruments, click **<Detect>**.

In-situ Audiometry

The In-situ Audiometry tool lets you measure the client's hearing using the hearing instruments as transducers. This means that the results will be specific for the instrument and acoustic coupling.

Instrument Acoustics

Here you can select hearing instruments for simulation or detect hearing instruments. You can also detect hearing instruments in all screens via the detect button in the menu bar.

Acoustic dashboard

After the hearing instrument is selected or connected, the “acoustic dashboard” is shown. There you can enter the acoustic options.

The acoustic dashboard also shows updated information about the hearing instrument performance with the selected acoustic options. It can be opened from any screen during the session to view or change the acoustic options. The graph view in the dashboard can be configured in the preference menu.

Fit Instrument

This menu is used to fit the product to the hearing loss of the client. In this menu, fitting tools such as Adaptation, Fine Tuning, Manage Programs, Feedback Manager and Frequency Lowering can be found. Depending on the product, many parameters of the product features (Directionality, Noise Reduction, etc.) can be set.

Transfer Settings

This tool lets you transfer gain, MPO and Adaptation to a new selected hearing instrument in a fitting session. These settings are copied – as closely as possible, given the limitations of the target hearing instrument – and adjusted to the new acoustic set-up. All other settings remain as prescribed for the target hearing instrument. Transfer Settings is not intended to replace a standard fitting.

Feature Selection

This menu offers screens to adjust the hearing instruments' general settings and program specific features.

For details about the audiological features of our products, please refer to the product information or our training material.

Finish session

This menu is used to configure local controls and indicators, to save and quit your fitting session in the office system database, and to save the data in the hearing instrument.

A copy of the hearing instrument settings is preserved at the beginning of the fitting session and will be saved back to the hearing instruments if you decide to end your fitting session without saving changes.

Connection status indicator

It informs you about the connection status of the hearing instruments to the programming device at any time during the fitting session.

The indicator can show the following connection states:

- moving dashed lines indicate connection and pairing in progress;
- a solid green line indicates hearing instruments are connected;

- moving dashes in blue and red indicate data transfer in progress.

If the connection to the hearing instruments is interrupted, a conflict dialog box will inform you. In this case, you can continue in simulation mode and connect the hearing instruments later or try to re-connect to the hearing instruments.

For further guidance regarding HearSuite, please refer to our training material and the help file.

5. Philips Firmware Updater

You can start this updater from the Windows start menu or from HearSuite. Make sure that your computer is online. When you update hearing instruments, make sure that your patient is not wearing the hearing instruments while updating. Connect the instruments with a cable connection. Do not disconnect the instruments while updating.

After the update, the instruments will be set back to factory settings. Restore the client's specific settings before giving the hearing instruments back to the client and pair them with your client's remote control. Provide your client with the latest version of the "Instructions for Use" booklet.

For a detailed guide about the Firmware Updater, please refer to our training material and the HearSuite help file.

6. HearSuite Updater

With the updater you can download updates and new releases when they are available. It opens when you start Windows and checks for new updates. Make sure you set the preferences correctly so that the software is not unintentionally downloaded and installed at an inconvenient time.

For a detailed guide about the updater, please refer to our training material and the HearSuite help file.

7. Warnings

To avoid possible client injury or hearing instrument failure, for your personal safety and to ensure correct usage, you should familiarize yourself fully with the following general warnings before using the fitting software. Contact your local distributor if you experience unexpected operations or serious incidents with the fitting software during use or because of its use. Serious incidents should also be reported to the national authorities.

General safety precautions

For safety reasons, always read the Intended Use of the fitting software.

Connection

Take care not to lose the wired or wireless connection to the hearing instrument or let communication errors interrupt the fitting flow.

Choking hazards

For safety reasons, caution must be taken when fitting children younger than 36 months. Children younger than 36 months must always use a tamper-resistant battery drawer. For safety reasons, always use ear molds when fitting children younger than 36 months.

Firmware

For safety reasons, always ensure that the client is not wearing the hearing instrument during a firmware update, as the updated hearing instrument will be reset to factory settings after the update. Moreover, do not give the hearing instrument to the client before restoring user settings.

Ensure that the serial number of the connected hearing instrument corresponds to the serial number registered in the office automation system for the relevant client. Always note the firmware version of the hearing instrument before and after a firmware update.

Ensure that the firmware version of the programming device is compatible with the firmware version of the fitting software.

Avoid interruptions or connection failure to the hearing instrument or connectivity device during a firmware update.

REM system

Pay attention to any error messages from the REM system caused by incorrect or unintended data sent to the REM system.

Power instruments

For safety reasons, caution should be taken when selecting, fitting and using a hearing instrument where the maximum sound pressure capability exceeds 132 dB SPL (IEC 711) as there may be a risk of damaging residual hearing of the hearing instrument user.

Transfer Settings

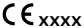
Your client must not wear the instruments while transferring settings.

In-situ Audiometry

In-situ Audiometry should not be used for diagnostic purposes. Make sure that you add an audiogram in NOAH or the Client step in HearSuite before you conduct In-situ Audiometry.

Remove any Real Ear Fit results before you conduct In-situ Audiometry.

Make sure you carry out In-situ Audiometry in a quiet area.

Symbol	Description
	<p>Manufacturer The product is produced by the manufacturer whose name and address are stated next to the symbol. Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.</p>
	<p>CE mark The product complies with Medical Device Directive 93/42/EEC. The four-digit number indicates the identification of the notified body.</p>
	<p>Warnings Text marked with a marking symbol must be read before using the product.</p>
	<p>Electronic waste (WEEE) Recycle hearing instruments, accessories or batteries according to local regulations. Hearing instrument users can also return electronic waste to their hearing care professional for disposal. Electronic equipment covered by Directive 2012/19/EU on waste and electrical equipment (WEEE).</p>
	<p>Consult the Instructions for Use</p>
	<p>Medical Device Symbol</p>

8. Disclaimer

The manufacturer does not assume any liability for compliance with local requirements regarding fitting of hearing instruments. It is entirely your responsibility as the user of HearSuite to ensure that you have the required training and/or authorization for fitting hearing instruments. The manufacturer does not take responsibility for the consequences of using the fitting software outside intended use or warnings.

2020

Manufactured by:

SBO Hearing A/S
Kongebakken 9
DK-2765 Smørum

Canada:

Imported and distributed by:

Audmet Canada Ltd
1600 – 4950 Yonge St
Toronto, ON M2N 6K1

©2020, SBO Hearing A/S
All rights reserved.

CE 0543

hearingsolutions.philips.com

Philips and the Philips Shield Emblem are registered trademarks of Koninklijke Philips N.V. and are used under license. This product has been manufactured by or for and is sold under the responsibility of SBO Hearing A/S, and SBO Hearing A/S is the warrantor in relation to this product.

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