

Oticon Medical Streamer Bimodal Guide


A patient wearing a wireless Ponto sound processor on one ear and a select super power Oticon hearing aid* on the other ear can receive information from the same Oticon Medical Streamer.




Oticon Medical Streamer comes OPEN, meaning that it works directly out of the box with the **Oticon hearing aid**. We recommend that the Streamer remains OPEN. You don't need to take any special steps, it is ready to use in this mode with both devices.



End Fitting

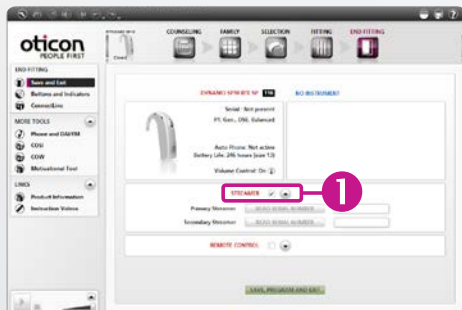

End Fitting

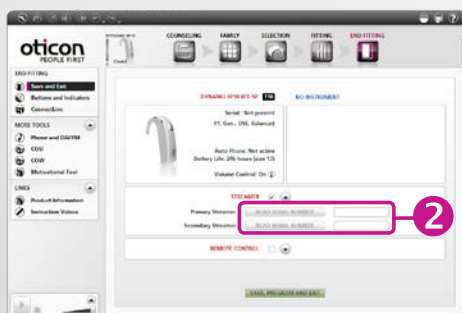

If you would like to link Oticon Medical Streamer anyways, you must do so in Oticon Genie:

1. Select the Streamer in the End Fitting step
2. Enter the Oticon Medical Streamer serial number by either:

Using NOAHlink or EXPRESSlink:

Place the streamer close to the hearing aid, and click "READ SERIAL NUMBER"

Using HI-PRO:

Enter the serial number manually. It can be found on the bottom of the Streamer or on the Oticon Medical Streamer box.

3. Click Save, Program and Exit

* See www.oticonmedical.com/OMStreamerHA for a complete list of Oticon hearing aids that work with Oticon Medical Streamer.

If you decide to link OM Streamer to the Oticon hearing aid* you must also link OM Streamer to the wireless Ponto sound processor. Turn the page to complete the linking process.

Oticon Medical Streamer Guide



Oticon Medical Streamer comes OPEN, meaning that it works directly out of the box with the wireless Ponto sound processor. We recommend that the Streamer simply remains OPEN. You don't need to take any special steps, it is ready to use in this mode with the patient's wireless Ponto sound processor(s).



End Fitting



If you would like to link Oticon Medical Streamer anyways, you must do so in Genie Medical:

1. Select Streamer in End Fitting step

2. Click ConnectLine

ConnectLine



3. Click Settings

4. Connect the Oticon Medical Streamer to the PC



5. Select Linked

6. Close Streamer Settings

Streamer Settings



7. Click End Fitting step

8. Enter the Oticon Medical Streamer serial number by either:

Using NOAHlink or EXPRESSlink:

Place the streamer close to the wireless Ponto sound processor, and click "READ SERIAL NUMBER".

Using HI-PRO:

Enter the serial number manually. It can be found on the bottom of the Streamer or on the Oticon Medical Streamer box.

End Fitting



9. Click Save, Program and Exit.