

# SnapPROBE™ for easyScreen

New OAE + ABR ear probe designed specifically for infants

## Features at a Glance

- Uncompromised acoustic performance for both OAE and ABR hearing screening
- Easy eartip installation and removal
- Minimal cleaning required, saving time
- Disposable front end enhances infection control
- Wax guards protect sensitive components for improved durability

The SnapPROBE™ achieves these by a completely disposable front end, the tri-lumen eartip, which traps earwax before it reaches the sensitive electronics in the probe body. Minimal cleaning is required. Just snap off the entire front end and throw it away. The chance of cross-contamination is significantly reduced since debris rarely reaches the probe body. If excess wax or debris from the ear canal does travel down the eartip openings, then there is a second line of protection. Borrowing from popular hearing aid technology, the SnapPROBE features replaceable wax guards that protect the speakers and microphone.

Installing the eartip is also easy. Just push it straight onto the probe body. Because the design is symmetrical, it will find its way onto the probe body channels. To remove the eartip just snap it off the probe body and throw it away.

**easyScreen firmware version 1.16.21 installation required in order to use the SnapPROBE.**

Supported tests	DPOAE, TEOAE and monaural ABR
Cable length	120 cm / 47 in
Memory	Calibration values and transducer ID
Eartip	Replaceable, with 3 acoustic channels
Weight (incl. cables)	20 g / 0.71 oz



## MAICO Diagnostics

10393 West 70th Street  
Eden Prairie, MN 55344, USA

Tel.: (888) 941-4201  
Fax: (952) 903-4100

info@maico-diagnostics.com  
www.maico-diagnostics.com/us