


How to pair accessories


Pairing your ConnectClip with your instrument(s)

1. Ensure ConnectClip is powered off.

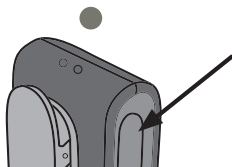


Press and hold the multi-button for approx. 6 seconds until power indicator OFF ●

2. Turn instrument(s) OFF.

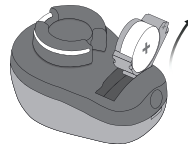

(open battery drawer)


3. Turn instrument(s) ON.

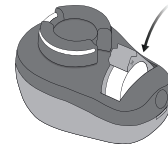

(close battery drawer)


4. The instrument(s) will now be ready for a first time connection with the ConnectClip for about 3 minutes

5. Turn ON ConnectClip.

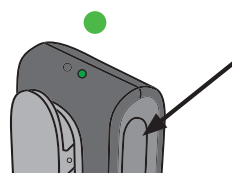


Press and hold the multi-button for approx. 3 sec. until power indicator STEADY Green ●

6. ConnectClip will now search for your instrument(s).

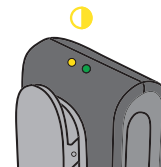


Status indicator BLINKING YELLOW ●

7. Place ConnectClip and instrument(s) close to each other.

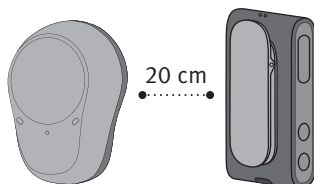

(max distance 20 cm)

8. Connection established with instrument(s)

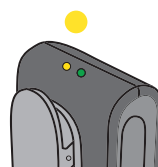

is indicated by status indicator STEADY YELLOW ●

How to pair accessories

Pairing your EduMic with your instrument(s)

1. Ensure EduMic is turned on.




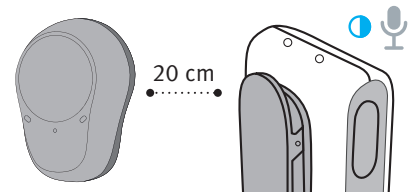
2. Set the instrument(s) in pairing mode.

Restart the instrument(s) to activate pairing mode.
Read your instruments' Instructions For Use to see how to turn the instrument(s) off and on again


3. The instrument(s) will now be ready for a first-time connection with EduMic within the next 3 minutes

4. Pair the instrument(s).


Place the instrument(s) close to EduMic. It is recommended to pair one instrument or set of instruments at a time. The status indicator will flash BLUE  while pairing is in progress



5. Connection established with Instrument(s).

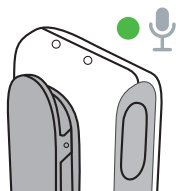


EduMic is now ready for use with the instrument(s). If more sets of instruments need to be paired, repeat these steps

The status indicator will stop flashing BLUE when pairing is complete and turn GREEN 

How to pair accessories


Pairing your iPhone with your instrument(s)

1. Settings.

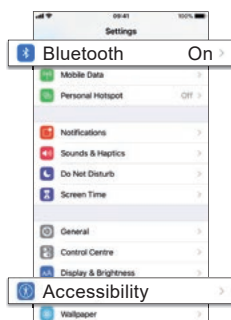


Go to your iPhone settings. Check to make sure that Bluetooth is on. Then choose “Accessibility”

2. Accessibility.

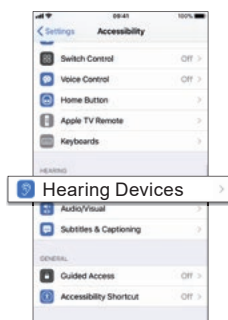


On the “Accessibility” screen, choose “Hearing Devices”

3. Prepare.

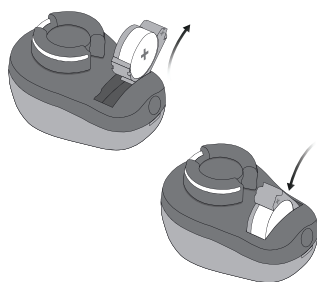


Open and close the battery drawer on the instrument(s), and place them close to your iPhone. The instrument(s) remains in pairing mode for 3 minutes

4. Select.

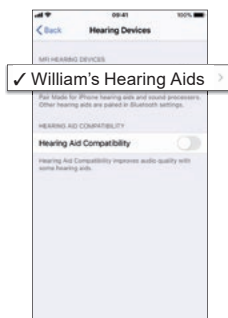

Your iPhone will detect the instrument(s) for pairing. Detected devices will appear in your iPhone list. Choose your instrument(s)

5. Confirm pairing.

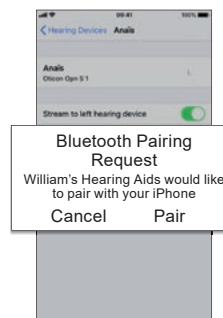


Confirm pairing. If you have two instruments, pairing confirmation is needed for each instrument

How to pair accessories

Pairing your remote control with your instrument(s)

1. Prepare remote control.

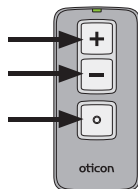


Press and hold all buttons for 5 seconds. The status LED will flash green twice

2. Confirms ready for pairing.




The status LED will then glow green for about 10 seconds to confirm the remote is ready for pairing


3. Turn the instrument(s) OFF. (open battery drawer)

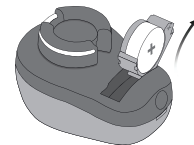

4. Turn the instrument(s) ON. (close battery drawer)

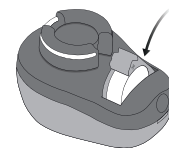


5. Place Remote Control and the instrument(s) close to each other (max distance 20 cm).

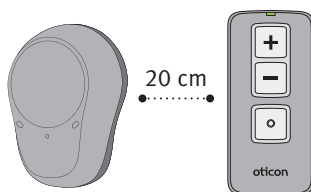

The instrument(s) will be ready for a first time connection with Remote Control for about 3 minutes

6. Pair the remote control with the instrument(s).

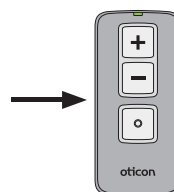

Briefly press any key on the remote control to make it automatically pair with the instrument(s)

7. Connection established with instrument(s).




When pairing is complete the status light glows green for about 10 seconds


How to pair accessories

Pairing your TV Adapter with your instrument(s)

1. Make sure the TV Adapter is powered on.

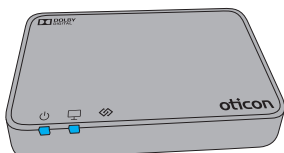


2. Turn instrument(s) OFF.

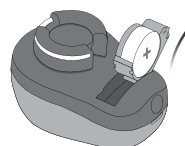

(open battery drawer)


3. Turn instrument(s) ON.

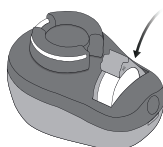


(close battery drawer)

4. The instrument(s) will now be ready for a first time connection with the TV Adapter for about 3 minutes

5. Place the instrument(s) close to or on top of the TV Adapter.

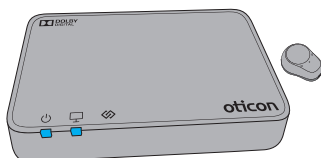

6. Connection established with instrument(s).

TV Adapter	Description
	Pairing completed

Pairing is completed when the power indicator and the TV indicator turn blue for about 30 sec.